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**AcuJect® / Cook Endoscopy / Marcon-Haber**

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**AcuJect® / Cook Endoscopy / Marcon-Haber**

16386/0106

**COOK®**

**Endoscopy**

English . . . . .	1
中文/Chinese . . . . .	4
Dansk/Danish . . . . .	6
Nederlands/Dutch . . . . .	9
Français/French . . . . .	12
Deutsch/German . . . . .	15
Ελληνικά/Greek . . . . .	18
Italiano/Italian . . . . .	22
Português/Portuguese . . . . .	25
Español/Spanish . . . . .	28
Svenska/Swedish . . . . .	31

## **INTENDED USE**

Cook Endoscopy Disposable Varices Injectors are used for endoscopic injection into gastrointestinal mucosa. The Disposable Varices Injectors are available with a Luer slip handle or a thumb ring handle. These devices are supplied sterile and intended for single use only.

## **NOTES**

Do not use this device for any purpose other than the stated intended use.

Inventory rotation of sterile products is essential. Verify the expiration date on the package label prior to using the product. If the expiration date has lapsed, do not use or resterilize the device.

If the product package is open or damaged when received, do not use this device.

Cook Endoscopy devices must be stored in a dry location, away from temperature extremes.

## **CONTRAINDICATIONS**

Contraindications include those specific to the primary endoscopic procedure to be performed in gaining access to the desired injection site.

Severe coagulopathy.

## **POTENTIAL COMPLICATIONS**

Potential complications associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Potential complications associated with mucosal injection include, but are not limited to: ulceration or necrosis of injected tissue, venous thrombosis, stricture formation.

Other potential complications to be considered if injection is performed in the esophagus include, but are not limited to: chest pain, mediastinitis, pleural effusion.

## **PRECAUTION**

The coordination of the endoscope accessory channel size with compatible devices is essential in order to obtain optimal results during a procedure. Cook Endoscopy Disposable Varices Injectors are 5.0 FR or 7 FR and require a minimum channel size of 2.0 mm or 2.8 mm respectively.

## **CAUTIONS**

A thorough understanding of the technical principles, clinical applications and risks associated with GI endoscopy and mucosal injection is

necessary before using this device. Cook Endoscopy Disposable Varices Injectors should only be used by, or under the supervision of, physicians thoroughly trained in therapeutic endoscopy and mucosal injection. The needle must be retracted into the sheath prior to introduction, advancement, or withdrawal of the device from the endoscope. Failure to retract the needle may result in damage to the endoscope.

## PRODUCT INSPECTION

1. Upon removing the device from the package and protective holder, visually inspect with particular attention to kinks, bends, or breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook Endoscopy for return authorization.
2. Uncoil the catheter and straighten completely.
3. Twist Luer slip fitting to disengage it from the adjustment wheel prior to adjusting the needle extension. (*See fig. 1*) **Caution:** Attempting to adjust the needle while the Luer slip fitting is engaged may result in damage to the device. **Note:** The Disposable Varices Injectors have a 1 cm black band at the tip of the catheter to aid in needle visualization during mucosal injection. This band is not shown in the illustrations in order to demonstrate the positions of the needle. **Note:** The adjustment wheel assembly for all of the Disposable Varices Injectors is the same as the Variable Injection Needle shown in figures 1-5.
4. To ensure desired needle extension, advance the Luer slip fitting and engage it firmly into the adjustment wheel. (*See fig. 2*)
5. To adjust the needle, retract the needle into the outer sheath. (*See fig. 3*)
6. Hold the device with the Luer lock fitting facing the operator and rotate the adjustment wheel clockwise to lengthen the needle extension, counterclockwise to shorten the needle extension. (*See fig. 4*)
7. Verify the adjustment by advancing and engaging the Luer slip fitting. (*See fig. 5*) The needle should move in and out of the outer sheath easily. Inspect the needle for breaks. **Caution:** Advancing and retracting the needle while the catheter is coiled may result in stretching of the catheter and damage to the performance characteristics of the device.

## INSTRUCTIONS FOR USE

1. Securely attach a pre-filled syringe to the Luer lock fitting on the device, then flush air from the needle. Check for patency and leaks. **Caution:** Prior to introducing the device into the accessory channel of the endoscope, twist Luer slip fitting to disengage and retract the needle into the sheath to avoid puncturing the channel wall of the endoscope.
2. With the needle fully retracted into the sheath, introduce the device into the accessory channel of the endoscope.

3. Advance the device in short increments until it is visualized exiting the endoscope.
4. Endoscopically visualize the desired injection site and maintain the position of the endoscope.
5. With the sheath in endoscopic view, advance and secure the needle by firmly engaging the Luer slip fitting into the adjustment wheel. **Note:** If desired, the needle extension may be adjusted by withdrawing the needle into the sheath and fine tuning the needle extension using the adjustment wheel. **Caution:** During needle adjustment or advancement, ensure the sheath of the device has exited the endoscope completely and is within endoscopic view.
6. Advance the needle into the mucosa, then using the attached syringe, perform injection.
7. When injection is complete, withdraw the needle from the injection site and twist Luer slip fitting to disengage and retract it into the sheath by pulling back on the Luer slip fitting. Repeat the procedure as necessary.
8. Upon completion of the procedure, twist Luer slip fitting to disengage and retract the needle into the sheath, then remove the device from the endoscope.
9. Dispose of the device per institutional guidelines for biohazardous medical waste.

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AcuJect is a registered trademark of Wilson-Cook Medical Inc.
Cook is a registered trademark of Cook Incorporated.
**Cook Ireland Ltd.**