**Colonic Z-Stent® Endoprosthesis System**  
**Colonic Z-Stent® Sistema de Endoprótese**  
**Sistema de endoprótesis Z-Stent® para colon**

18840/0105

**COOK**® Wilson-Cook Medical  
GI Endoscopy

English . . . . .	1
Português . . . . .	5
Español . . . . .	10

## **INTENDED USE**

The Colonic Z-Stent is a self-expanding, tubular prosthesis used to maintain patency of malignant colonic strictures in patients having high operative risk and/or advanced disease. The Wilson-Cook Colonic Z-Stent Introduction System is used to place the Colonic Z-Stent. This device and introduction system are supplied sterile and intended for single use only.

## **STENT DESCRIPTION**

The Colonic Z-Stent is constructed of a series of 2 cm cages, depending on the length of the stent. When fully expanded, the stent forms a tubular shape 25 mm in diameter and is flared at both ends to allow anchoring within the colonic wall. The stent is available in 4, 6, 8, 10, and 12 cm lengths with a flare diameter of 35 mm.

## **INTRODUCTION SYSTEM DESCRIPTION**

The Wilson-Cook Colonic Z-Stent Introduction System consists of a radiopaque inner catheter, an outer sheath with graduated markings, a guiding catheter, a radiopaque dilator tip, a loading funnel, a safety locking key and a 200 cm Savary-Gilliard® wire guide.

## **NOTES**

Do not use this device for any purpose other than the stated intended use. If the product package is open or damaged when received, do not use this device. Wilson-Cook devices must be stored in a dry location, away from temperature extremes

## **CONTRAINDICATIONS**

Contraindications include those specific to lower GI endoscopy.

Contraindications to the placement of the Colonic Z-Stent include, but are not limited to: total colonic obstruction, strictures that cannot be dilated to 10 mm, placement requiring positioning of the Z-Stent below the dentate line, surgical resection candidates, and benign disease.

Relative contraindications include, but are not limited to: uncooperative patient, coagulopathy, recent myocardial infarction, non-obstructing tumor, necrotic colonic mucosa and acutely angled stenosis.

## **POTENTIAL COMPLICATIONS**

Potential complications associated with lower GI endoscopy include, but are not limited to: perforation, hemorrhage, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Additional complications that can occur with the placement of a colonic prosthesis include, but are not limited to: stent misplacement, stent migration, tumor ingrowth/overgrowth, occlusion and pressure necrosis/erosion of the luminal mucosa.

## PRECAUTIONS

A complete diagnostic evaluation must be performed prior to use to determine proper Z-Stent size.

Prior to loading the Z-Stent, ensure that the loading string is not twisted.

## CAUTIONS

A thorough understanding of the technical principles, clinical applications and risks associated with lower GI endoscopy and colonic stent placement is necessary before using this device. The Colonic Z-Stent should only be placed by, or under the supervision of, physicians thoroughly trained in the procedure.

The Colonic Z-Stent should be placed using fluoroscopic monitoring.

The Colonic Z-Stent should only be placed with the Wilson-Cook Colonic Z-Stent Introduction System. The introduction system is provided with sets or may be purchased separately.

Prior to advancing the Colonic Z-Stent Introduction System into the colon, the area to be stented should be dilated to a minimum of 10 mm and a maximum of 20 mm. If the area is dilated greater than 20 mm, the stent may migrate.

The Colonic Z-Stent is intended for palliative treatment only. Alternate methods of therapy should be investigated prior to placement.

After Colonic Z-Stent placement, alternate methods of treatment such as chemotherapy, irradiation and Nd:Yag Laser, **should not** be administered as this may increase the risk of stent migration due to tumor shrinkage, or stent erosion and/or mucosal bleeding.

A minimum stent length of 4 cm longer than the lesion is recommended to bridge the stricture with 2 cm (1 cage) above and 2 cm (1 cage) below the affected area.

Long term patency with this stent has not been established. Periodic evaluation is advised.

## WARNINGS

The Colonic Z-Stent is **not intended to be removed**. Attempts to remove the stent after placement may cause damage to the mucosa. The Colonic Z-Stent is considered to be a permanent implant.

MRI compatibility of the Colonic Z-Stent has not been established.

## RECOMMENDED EQUIPMENT

- Quantum TTC® Colonic Balloon Dilators

## PATIENT PREPARATION

1. A complete diagnostic evaluation must be performed prior to placement to determine the proper Z-Stent length.

2. Upon removing the Z-Stent and introduction components from the packages, visually inspect the devices with particular attention to kinks, bends, and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook for return authorization.
3. Dilate the stricture to a minimum of 10 mm and a maximum of 20 mm using a Quantum TTC Colonic Balloon Dilator.
4. Endoscopically locate and mark the upper and lower margins of the lesion with a radiopaque marker. **Caution:** Accurate marking of the lesion borders is essential for proper Z-Stent placement. **Note:** Water-soluble markers will dissipate quickly.
5. Place the Savary-Gilliard wire guide, floppy tip first, through the endoscope accessory channel until the tip is fluoroscopically visualized as long as possible beyond the narrowed segment intended for stenting. Leave the wire guide in place and remove the endoscope.

## SYSTEM PREPARATION

1. Ensure the guiding catheter fitting is completely snapped into the fitting at the end of the inner pusher.
2. Apply the enclosed tube of water-soluble lubricant to the inside of the loading funnel flare and the inner rim of the outer sheath. **Note:** Prior to loading the Z-Stent, ensure the loading string is not twisted.
3. Load the stent into the introducer by grasping the funnel and stent in one hand and pulling the thumb-ring with the other hand. Continue pulling the thumb-ring until the stent is fully loaded into the introducer. **Note:** The stent should be guided into the funnel, if necessary, to facilitate the loading into the introducer.
4. After the stent is fully loaded into the outer sheath, disengage the loading funnel from the outer sheath and remove the loading funnel from the guiding catheter.
5. Gently push into the outer sheath any part of the stent that is not yet fully loaded. Pull the guiding catheter fitting to disengage it from the inner catheter. Advance the dilator tip into the outer sheath. Dilator tip should fit securely within the outer sheath. The tip will advance the stent approximately 2 cm into the outer sheath.
6. Place the end of the previously positioned wire guide through the guiding catheter. Gently advance the assembled system over the wire guide until the wire guide exits the fitting on the guiding catheter.

## INSTRUCTIONS FOR PLACEMENT

1. Using fluoroscopic guidance, advance the assembled introduction system over the pre-positioned Savary-Gilliard wire guide. Using the

collapsed cages as reference points, fluoroscopically locate the stent in the marked lesion. A stent length allowing 2 cm (1 cage) above and 2 cm (1 cage) below the stricture is appropriate.

2. Confirm the desired position fluoroscopically. While maintaining the introducer position, remove the locking key.
3. Fluoroscopically verify that the stent position within the lesion has been maintained. Reposition the introducer system as necessary.
4. Using continuous fluoroscopic monitoring, **SLOWLY** deploy the **first** cage of the stent by holding the inner catheter in place while **slowly** withdrawing the outer sheath. **Caution:** Failure to deploy the stent **slowly** may cause improper positioning. **Note:** During deployment of the first cage, the stent position may be adjusted by **slowly** advancing or withdrawing the entire introduction system. Continue deployment, one cage at a time, by **slowly** withdrawing the outer sheath, until the stent is completely deployed.
5. Cut the suture attached to the thumb-ring and remove the thumb-ring, leaving the suture in place through the stent.
6. Withdraw the introduction system, leaving the loading suture in place through the stent. **CAUTION:** If the loading suture becomes compressed against the guiding catheter and outer sheath during removal of the introduction system, reengage the guiding catheter into the inner catheter of the sheath to free the suture. Do **not** remove the introduction system while the suture is caught, as this may cause inadvertent displacement of the stent.
7. Introduce the endoscope and advance to the upper margin of the stent in order to endoscopically confirm position and patency of the stent. **Caution:** Do not introduce the endoscope into the stent, as displacement may occur. **Caution:** Tumor impingement may prevent the stent from immediately achieving the maximum 25 mm diameter. A Quantum TTC dilation balloon may be used to expand the stent to 25 mm. **Note:** The loading suture may be used as an aid in stent repositioning, if necessary.
8. After proper stent position is verified endoscopically, continue to endoscopically monitor the stent while gently pulling on one end of the loading suture until the suture is completely removed.
9. Upon completion of the procedure, dispose of the introduction system components per institutional guidelines for biohazardous medical waste.

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Z-Stent, Savary-Gilliard and Quantum TTC are registered trademarks of Wilson-Cook Medical Inc.
Cook is a registered trademark of Cook Incorporated.
**Wilson-Cook Medical Inc.**
**Cook Ireland Ltd.**