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INTENDED USE

The Balloon Replacement Gastrostomy Tube is intended to be used as a replacement gastrostomy tube through an established gastrostomy tract. This device provides continued access to the stomach for delivery of nutrition and medication to patients requiring nutritional support. This system also allows decompression as prescribed by the physician. This device is supplied sterile and intended for single use only.

CONTENTS

- 1 Balloon Replacement Gastrostomy Tube
- 1 Twist Lock
- Gauze Pads
- 1 Prefilled 10 cc Luer Syringe (12 to 16 FR)
- 2 Prefilled 10 cc Luer Syringes (18 to 28 FR)
- 2 pks. Water-Soluble Lubricant (USA only)
- 1 Instruction Booklet
- 1 Patient Care Manual

NOTES

Do not use this device for any purpose other than the stated intended use.

Inventory rotation of sterile products is essential. Verify the expiration date on the package label prior to using the product. If the expiration date has lapsed, do not use or resterilize this device.

If the product package is open or damaged when received, do not use this device.

Wilson-Cook devices must be stored in a dry location, away from temperature extremes.

CONTRAINDICATIONS

Contraindications associated with placement and use of a Balloon Replacement Gastrostomy Tube include, but are not limited to: patient without an established gastrostomy tract, signs of infection such as redness, edema or purulent drainage present, sepsis, severe gastroesophageal reflux, large ascite, or diffuse inflammatory, infectious, or neoplastic disease involving the walls of the abdomen or anterior stomach, gastrointestinal obstruction or proximal small bowel fistulae.

POTENTIAL COMPLICATIONS

Potential complications associated with placement and use of a Balloon Replacement Gastrostomy Tube include, but are not limited to: bronchopulmonary aspiration and pneumonia, respiratory distress or airway obstruction, peritonitis or septic shock, colocutaneous, gastrocolocutaneous or small bowel fistula, gastric dilatation, sigmoid

intra-abdominal herniation and volvulus, persistent fistula following tube removal, gastric dilatation, esophageal injury, necrotizing fasciitis, candida cellulitis, improper placement or inability to place tube, tube dislodgment or migration, hemorrhage, and tumor metastasis.

Additional complications include, but are not limited to: pneumoperitoneum, peristomal wound infection and purulent drainage, stomal leakage, bowel obstruction, gastroesophageal reflux (GERD), and blockage or deterioration of the balloon replacement tube.

PRECAUTIONS

The benefits of a balloon replacement tube to the patient must be weighed against the risks associated with any indwelling gastrostomy feeding tube.

During placement and use, care must be taken to avoid cutting, crimping, or damaging components.

Do not modify the balloon replacement tube in any way.

Follow the instructions and the Patient Care Manual supplied with each kit. The Patient Care Manual is intended as a reference for the caregivers of the patient. It is essential for the Patient Care Manual to accompany the patient and be explained to all people responsible for the care of the patient. Additional Patient Care Manuals are available from Wilson-Cook Medical Inc.

All PEG Systems, including the Balloon Replacement Gastrostomy Tube, are radiopaque. Proper location and integrity of any internal component can be visualized by x-ray.

The Balloon Gastrostomy Tube must be inflated with **sterile water only**.

Do **not** use air or saline to inflate the balloon, as they may cause premature deflation.

Manufacturer guidelines are not intended to replace physician's recommendations.

CAUTIONS

A thorough understanding of the technical principles, clinical applications and risks associated with placement and/or removal of a Balloon Replacement Tube is necessary before using this device. Placement and/or removal of the Balloon Replacement Tube should only be performed by, or under the supervision of, physicians thoroughly trained in the procedure.

When placing a balloon replacement tube in obese patients, all anatomical structures must be identified prior to placement.

Replacement of the Balloon Replacement Gastrostomy Tube is recommended every 29 days or at the discretion of the Health Care Provider.

WARNINGS

Do not over inflate the balloon. Feeding into an over inflated balloon may result in damage to the gastric mucosa, tube migration, tube failure, sepsis, infection or death.

Excessive traction on the feeding tube may cause premature removal, fatigue or failure of the device.

INSTRUCTIONS FOR USE

Tube Placement

1. Upon removing the device and its components from the package, visually inspect with particular attention to kinks, bends or breaks in the feeding tube assembly. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook for return authorization.
2. Evaluate the existing stoma tract to ensure a well-established gastrostomy tract into the stomach.
3. Push the pivotal bolster above the 10 cm mark.
4. Fill the enclosed Luer lock syringe with sterile water. **Note:** Refer to the adjacent chart for the appropriate amount of sterile water for each balloon replacement tube.
5. Generously lubricate the tip of the replacement tube and the stoma site with **water-soluble lubricant**.
6. Gently advance the tube into the stoma site until the balloon is completely in the stomach. (*See fig. 1*)
7. Securely attach the pre-filled syringe to the access port on the balloon replacement tube labeled "BALLOON" and inflate the balloon. (*See fig. 2*) **Warning:** Do not overinflate the balloon. Feeding into an over inflated balloon may result in damage to the gastric mucosa, tube migration, tube failure, sepsis, infection or death.
8. Withdraw the tube gently until tension is felt as the balloon rests against the stomach wall. (*See fig. 3*)
9. Slide the pivotal bolster down the shaft of the tube until the bolster rests gently against the abdominal wall. **Warning:** The bolster should rest gently on the skin surface. Excessive traction on the tube may cause premature removal, fatigue, or failure of the device.
10. Secure the twist lock around the bolster collar, being careful not to crimp it. (*See fig. 4*) **Important:** Use the twist lock to secure the bolster

BALLOON REPLACEMENT SIZE CHART

French Size	Inflated
	Balloon Volume (Sterile Water)
14	5 cc
18	20 cc
24	20 cc

to the tube. This will help to prevent future migration of the tube and reduce the need to constantly reposition or pull on the tube.

11. Dispose of any unused items per institutional guidelines for biohazardous medical waste.
12. Note the centimeter marking on the tube that is closest to the bolster and record it on the patient's chart and on the patient information sheet in the Patient Care Manual. **Note:** The Patient Care Manual enclosed in the kit is intended as a reference for the caregivers of the patient. It is essential for the Patient Care Manual to accompany the patient and be explained to all people responsible for the care of the patient.
13. Record physician's instructions for feeding and administration of prescribed medications in the Patient Care Manual.
14. The patient should remain NPO for 24 hours unless otherwise directed by a physician.

Tube Removal

Caution: If a replacement tube is desired, it must be placed immediately after removal. External tube removal may result in minimal trauma or bleeding that may require treatment.

The feeding tube may be removed in the following manner:

1. Attach a Luer lock syringe to the Balloon port on the tube and deflate the balloon completely. If the balloon does not deflate, cut the white tubing below the Balloon port. This will deflate the balloon completely.
2. Gently remove the tube from the stoma tract.
3. Dispose of the Balloon Replacement Gastrostomy Tube per institutional guidelines for biohazardous medical waste. The stoma tract should heal and close within 24 hours.

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Cook is a registered trademark of Cook Incorporated.
Cook Ireland Ltd.