The Web™ Extraction Basket

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18877/0105

COOK® Wilson-Cook Medical
GI Endoscopy

English	1
中文/Chinese	4
Português/Portuguese	6
Español/Spanish	9

INTENDED USE

The WEB Extraction Basket is used for the endoscopic removal of biliary stones and foreign bodies. The basket is constructed of multifilament Memory™ wire for shape retention and is compatible with the Soehendra® Lithotripter. This device is supplied sterile and intended for single use only.

NOTES

Do not use this device for any purpose other than the stated intended use.

Inventory rotation of sterile products is essential. Verify the expiration date on the package label prior to using the product. If the expiration date has lapsed, do not use or resterilize the device.

If the product package is open or damaged when received, do not use the device.

Wilson-Cook devices must be stored in a dry location, away from temperature extremes.

CONTRAINDICATIONS

Contraindications include those specific to the primary endoscopic procedure to be performed in conjunction with the removal of a foreign body.

Perforation.

If the WEB basket is to be used for biliary stone extraction, contraindications include an ampullary opening inadequate to allow for the unimpeded passage of the stone and basket.

POTENTIAL COMPLICATIONS

Potential complications associated with gastrointestinal endoscopy and ERCP include, but are not limited to: pancreatitis, cholangitis, sepsis, perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction, respiratory depression or arrest, cardiac arrhythmia or arrest.

Other potential complications associated with use of an extraction basket include, but are not limited to: impaction of the object, aspiration of foreign body, localized inflammation, pressure necrosis.

PRECAUTION

The coordination of endoscope accessory channel size with compatible devices is essential in order to obtain optimal results during a procedure. The sheath of the WEB basket is 7.0 French and requires a minimum channel size of 2.8 mm.

CAUTIONS

A thorough understanding of the technical principles, clinical applications, and risks associated with the specific type of endoscopy to be performed and with basket extraction is necessary before using this device. The WEB

baskets should only be used by, or under the supervision of, physicians thoroughly trained in the endoscopy indicated for the patient and in the use of endoscopic extraction baskets.

If this device is to be used for the removal of biliary stones, the following cautions apply:

- Assessment of the stone size and ampullary orifice must be made to determine the necessity of sphincterotomy. In the event sphincterotomy is required, all appropriate cautions, warnings, and contraindications must be observed.
- The WEB extraction baskets are compatible with the Soehendra Lithotripter.

Injection of contrast during ERCP should be monitored fluoroscopically. Overfilling of the pancreatic duct may cause pancreatitis.

The WEB extraction basket is **not** autoclavable.

The WEB baskets should never be coiled in less than an 8 inch (20 cm) diameter.

WARNINGS

When retrieving a foreign body from the upper GI tract, extreme caution must be exercised to avoid occlusion of the airway.

Surgical intervention may be required if impaction occurs.

INSTRUCTIONS FOR USE

1. Upon removing the device from the package, uncoil the device and visually inspect with particular attention to kinks, bends, or breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook for return authorization.
2. With the basket fully retracted into the sheath, insert the device into the endoscope accessory channel. Advance the device through the channel in short increments until the basket sheath exits the endoscope.
3. Endoscopically or fluoroscopically visualize the extraction target and advance the basket to the appropriate position. Fluoroscopic visualization may be enhanced by injecting contrast via the side arm fitting. Place the sheath of the basket beyond the target before extending the basket to entrap the extraction target.
4. Confirm the desired position of the basket sheath relative to the target. With one hand grasping the grips on the handle near the side arm fitting, gently push the handle forward with the other hand to advance the basket out of the sheath. (See fig. 1) **Caution:** Pulling on the sheath while advancing or retracting the basket may damage the device, rendering it inoperable.

5. Place the basket around the extraction target or foreign body and slowly retract the handle until entrapment is achieved. **Note:** It may be necessary to move the basket back and forth along the target in order to capture the object.
6. Withdraw the basket into the channel until the foreign body is supported against the endoscope tip, then withdraw the endoscope from the patient. Maintain endoscopic visualization to ensure continued entrapment of the object. **Caution:** If difficulty is encountered when removing the basket from the duct, moderate force may be applied by pulling on the handle to manually fracture the stone. If passage is still restricted, surgical intervention may be necessary.
7. Prepare or dispose of the extracted specimen per institutional guidelines.
8. Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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The WEB and Memory are trademarks of Wilson-Cook Medical Inc.
Soehendra and Wilson-Cook are registered trademarks of Wilson-Cook Medical Inc.
Cook is a registered trademark of Cook Incorporated.
Wilson-Cook Medical Inc.
Cook Ireland Ltd.