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INTENDED USE

The Gastro-Jejunal Feeding Tube is intended for placement through a percutaneous endoscopic gastrostomy tube to provide enteral nutrition to patients requiring nutritional support. This system also allows decompression as prescribed by the physician. This device is supplied sterile and intended for single use only.

CONTENTS

- 1 Gastro-Jejunal Feeding Tube
- 1 240 cm Wire Guide
- 1 Twist Lock
- 1 Cable Tie
- 1 Air Plug
- 2 pks. Water-Soluble Lubricant (USA only)
- 1 Instruction Manual
- 1 Patient Care Manual

NOTES

Do not use this device for any purpose other than the stated intended use.

Inventory rotation of sterile products is essential. Verify the expiration date on the package label prior to using the product. If the expiration date has lapsed, do not use or resterilize this device.

If the product package is open or damaged when received, do not use this device.

Wilson-Cook devices must be stored in a dry location, away from temperature extremes.

CONTRAINDICATIONS

Contraindications associated with placement and use of a PEG-J tube include, but are not limited to: sepsis, severe gastroesophageal reflux, large ascite, or diffuse inflammatory, infectious, or neoplastic disease involving the walls of the abdomen or anterior stomach, gastrointestinal obstruction or proximal small bowel fistulae.

POTENTIAL COMPLICATIONS

Potential complications associated with placement and use of a PEG-J tube include, but are not limited to: bronchopulmonary aspiration and pneumonia, respiratory distress or airway obstruction, peritonitis or septic shock, colcutaneous, gastrocolocutaneous or small bowel fistula, gastric dilatation, sigmoid intra-abdominal herniation and volvulus, persistent fistula following PEG-J removal, gastric dilatation, esophageal injury, necrotizing fasciitis, candida cellulitis, improper placement or inability to place PEG-J tube, tube dislodgment or migration, hemorrhage, and tumor metastasis.

Additional complications include, but are not limited to: pneumoperitoneum, peristomal wound infection and purulent drainage, stomal leakage, bowel obstruction, gastroesophageal reflux (GERD), and blockage or deterioration of the PEG-J tube.

PRECAUTIONS

The benefit of a PEG-J tube to the patient must be weighed against the risks associated with any indwelling gastrostomy feeding tube.

During placement and use, care must be taken to avoid cutting, crimping, or damaging components.

Do not modify the PEG, J-tube or adapters in any way.

Follow the instructions and the Patient Care Manual supplied with each kit. The Patient Care Manual is intended as a reference for the caregivers of the patient. It is essential for the Patient Care Manual to accompany the patient and be explained to all people responsible for the care of the patient. Additional Patient Care Manuals are available from Wilson-Cook Medical Inc.

All PEG Systems, including the PEG-J and FLOW-J, are radiopaque. Proper location and integrity of any internal component can be visualized by x-ray.

Proper positioning of the gastro-jejunal feeding tube must be verified by x-ray.

This device is only compatible with the appropriate Wilson-Cook PEG tube. Manufacturer guidelines are not intended to replace physician's recommendations.

CAUTIONS

A thorough understanding of the technical principles, clinical applications and risks associated with placement and/or removal of a PEG-J or FLOW-J tube is necessary before using this device. Placement and/or removal of the PEG-J tube should only be performed by, or under the supervision of, physicians thoroughly trained in the procedure.

When placing a PEG-J tube in obese patients, all anatomical structures must be identified prior to placement.

When placing a PEG-J tube, observe all institutional guidelines regarding gastroscopy, including removal of dentures.

PEG-J or FLOW-J replacement is recommended every **three** months or as recommended by the physician.

WARNING

Excessive traction on the feeding tube may cause premature removal, fatigue or failure of the device.

INSTRUCTIONS FOR USE

Tube Placement

1. Upon removing the device and its components from the package, visually inspect with particular attention to kinks, bends or breaks in the feeding tube assembly. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook for return authorization.
2. After introducing the gastroscope, insufflate the stomach and examine the mucosa. Determine if the mucosa is free of ulcerations or bleeding before proceeding. **Important:** This gastro-jejunal feeding tube (J-tube) is fully radiopaque and can only be used with the appropriate Wilson-Cook PEG tube. Ideally the PEG tube should be placed in the antrum or low in the body of the stomach to ease insertion of the J-tube.
3. Remove the feeding adapter and the twist lock or cable tie securing the external bolster from the previously placed Wilson-Cook PEG tube. The X mark on the PEG tube should be at the end. If not, the tube should be cut at the X mark. (See fig. 1)
4. Place a **non-spiked biopsy or alligator forceps** through the gastroscope and push the forceps through the PEG tube from the stomach until it is seen exiting the tube.
5. **Generously lubricate** the wire guide with **water-soluble lubricant**, especially at the tip.
6. Place the air plug over the wire guide and grasp the floppy tip end of the wire guide with the forceps. (See fig. 2)
7. Pull the wire guide and forceps back into the tube and place the air plug securely in the end of the PEG tube to minimize air leakage. (See fig. 3)
8. Insufflate the stomach to provide better visualization of the pylorus.
9. Advance the gastroscope with the wire guide and forceps through the pylorus and into the second portion of the duodenum.
10. **Straighten the wire guide by holding tension at both ends.** Advance the wire guide and forceps to the third portion of the duodenum.
11. **Generously lubricate** the J-tube **with water-soluble lubricant**, especially at the tip.
12. Remove the air plug from the G-tube, then place the J-tube over the wire guide. (See fig. 4)
13. While maintaining tension on the wire guide, slowly advance the J-tube over the wire guide. Continue advancement until it dislodges the forceps from the wire. **Note:** The J-tube should move distally when it dislodges the forceps from the wire.

14. Remove the forceps from the scope and plug the feeding adapter firmly into the PEG tube. (See fig. 5)
15. Endoscopically verify the J-tube position.
16. **Aspirate air, then remove the scope in a rotating fashion** to minimize the dragging effect on the J-tube.
17. Verify that the J-tube has not backed through the pylorus into the stomach. (See fig. 6) **Note:** A cable tie may be placed around the proximal gastrostomy tube at the feeding adapter to avoid inadvertent removal of the J-tube from the PEG.
18. Carefully remove the wire guide while observing endoscopically. Once again verify that the J-tube has not fallen back into the stomach, then remove the gastroscope.
19. Using a 60 cc piston irrigation syringe, flush 50 cc of water through the **yellow** J-tube port on the feeding adapter. The tube should flush easily if properly placed. **Note:** The red G-tube port on the feeding adapter is used to access the stomach.
20. Replace the twist lock or cable tie on the external bolster of the PEG tube, being careful not to crimp the tube. **Important:** Use the twist lock or cable tie to secure the pivotal bolster to the tube. This will help prevent future migration of the tube and reduce the need to constantly reposition or pull on the tube. **Warning:** Excessive traction on the gastric feeding tube may cause premature removal, fatigue or failure of the device.
21. Record physician's instructions for feeding and administration of prescribed medications in the Gastro-Jejunal Patient Care Manual.
22. Perform an abdominal x-ray following placement of the J-tube to verify proper tube position.

Tube Removal

Note: Prior to removal, we recommend the purchase of a new G-J tube, or universal adapter to plug into the Wilson-Cook PEG tube if only PEG tube feeding is desired.

Caution: If a replacement tube is desired, it must be placed immediately after removal. External tube removal may result in minimal trauma or bleeding that may require treatment.

G-J Tube Removal

1. Unplug the G-J adapter from the PEG tube, then gently pull the G-J tube to remove it completely from the PEG tube.
2. Dispose of the G-J tube per institutional guidelines for biohazardous medical waste.

3. To resume PEG feeding, place the universal adapter or bolus adapter into the PEG and resume feeding as prescribed.

PEG Tube Removal (External/Traction Method)

The PEG tube has been designed for removal using the external/traction method shown below. If this method is not possible, another method of removal such as endoscopic or surgical should be utilized.

The feeding tube may be removed without the need for endoscopy in the following manner:

1. Remove the twist lock or cable tie and bolster from the PEG tube.
2. Grasp the gastric feeding tube near the stoma site. While slowly rotating the tube, gently push 1-2 cm of the tube into the stomach to separate the tube from the stoma tract. **Warning:** If the tube does not rotate freely within the tract, do not attempt to use traction as a method of removal.
3. Hold the gastric tube near the stoma site and apply counter pressure by placing the fingers of the other hand around the base of the tube.
4. Loosely cover the stoma site with either a towel or a drape.
5. Holding the gastric feeding tube straight, apply steady traction to the tube until the internal dome emerges through the abdominal wall. **Caution:** The tube must be pulled straight out of the stoma tract.
6. Dispose of the feeding tube per institutional guidelines for biohazardous medical waste. The stoma tract should heal and close within 24 hours.

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Cook is a registered trademark of Cook Incorporated.
Cook Ireland Ltd.