Passport™
Low Profile Replacement Gastrostomy Device

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CONTAINS

Passport Low Profile Gastrostomy Device, Obturator, Gauze Pads, and Water Soluble Lubricant (USA only)

INTENDED USE

The Passport Low Profile Gastrostomy Device is used to replace a 20 French or larger gastrostomy tube that has been removed from an established stoma tract. This device provides continued access to the stomach for delivery of nutrition or medication or for gastric drainage or decompression. The Passport is supplied sterile and is intended for single use only.

NOTES

Do not use this device for any purpose other than the stated intended use.

Inventory rotation of sterile products is essential. Verify the expiration date on the Passport package label prior to using the product. If the expiration date has lapsed, do not use or resterilize the device.

If the product package is open or damaged when received, do not use this device.

Wilson-Cook devices must be stored in a dry location, away from temperature extremes.

CONTRAINDICATIONS

Contraindications to Passport insertion include, but are not limited to: gastric bleeding; gastric ulcers; an existing stoma tract less than the diameter of the selected Passport; anatomy resulting in a stoma tract greater than 4.4 cm or less than 1.2 cm in length; physical condition requiring jejunal feeding; a stoma tract which is tortuous, incomplete, immature or indeterminable.

Relative contraindications include, but are not limited to: a stoma tract that is irritated, granulated, infected, or actively bleeding.

POTENTIAL COMPLICATIONS

Potential complications associated with Passport placement include, but are not limited to: bleeding, tissue irritation, stoma site infection, leakage of gastric contents, peritonitis, gastrocolic fistula, sepsis and death.

CAUTIONS

A thorough understanding of the technical principles, clinical applications and risks associated with PEG removal and replacement is necessary before using this device. Placement of the Passport should only be performed by, or under the supervision of, physicians thoroughly trained in the procedure.

Accurate selection of the appropriate Passport size, as determined by the measurement of the removed gastrostomy tube, is essential. Refer to the Catheter Measuring Device Instructions.

Replacement of a removed gastrostomy tube or Passport must be accomplished as soon as possible. Most stoma tracts will close within 24 hours.

It is recommended that the patient be evaluated periodically for weight gain which may cause the existing Passport to be tight against the skin.

Passport replacement is recommended every five months or at the discretion of the physician.

WARNING

Introduction of any device into the stoma tract may result in disruption of the established tract. Complications that may result from disruption of the stoma tract include, but are not limited to: bleeding, tissue irritation, stoma site infection, leakage of gastric contents, peritonitis, gastrocolic fistula, sepsis and death.

INSTRUCTIONS FOR USE

1. Upon removal of the devices from the package, visually inspect with particular attention to kinks, bends or breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook for return authorization.
2. Cleanse the stoma site and examine the condition of the tissue after removing the existing gastrostomy tube. Evidence of skin irritation, bleeding, infection, or tissue granulation may require medical assessment prior to placement of the Passport.
3. Carefully determine the condition and direction of the stoma tract.
Warning: If the stoma tract is tortuous, incomplete, immature, or indeterminable, **do not** proceed with placement of this device. Seek endoscopic and/or fluoroscopic guidance to determine optimal tube replacement method.
4. Generously lubricate the stoma site and conical tip of the selected Passport using water soluble lubricant only. **Caution:** Use of non-water soluble lubricant may cause damage to the Passport.
5. Lubricate the obturator and insert it through the valve opening on the top of the device. Advance the obturator into the conical tip. *(See fig. 1)*
6. Continue advancing the obturator to elongate the tip. *(See fig. 2)*
7. Flatten the elongated conical tip against the obturator to aid insertion. *(See fig. 3)*
8. Using a slight twisting motion in the direction of the stoma tract, advance the elongated conical tip fully into the stomach.

Caution: If resistance is encountered, remove the obturator and Passport. Reassess to confirm the direction and verify the integrity of the stoma tract. Proceed cautiously with placement.

9. When the conical tip has been placed in the stomach, remove the obturator.

Note: The Passport should rest gently against the skin surface, with a minimum of 5 mm between the base of the external bolster and the skin surface.

Caution: If the Passport is tight against the skin, excessive pressure may cause leakage of gastric contents, edema, infection, tissue necrosis, and/or necrotizing fasciitis and should be replaced with a longer Passport.

10. Verify placement of the Passport by performing two tests.

- a. Apply gentle traction to the tube. **Note:** Resistance should be felt when traction is applied. If the Passport is improperly placed, gentle traction may cause the tube to dislodge.
- b. Gently rotate the Passport in the tract. **Note:** The Passport should rotate freely within the stoma tract.

Warning: If the Passport pulls out of the stoma tract or resistance is met when rotating the tube, the device may not be in the proper position. Remove the device completely by applying traction. Endoscopic or fluoroscopic guidance may be necessary for tube placement.

Caution: It is essential to verify proper position before accessing the Passport for the delivery of nutrition or medication or for gastric drainage or decompression. After verification of placement, feeding may begin immediately or at the discretion of the physician. Failure to verify proper positioning may result in complications including, but not limited to: infection, peritonitis, sepsis, and death.

11. A dressing is not recommended. If a dressing is desired, place it loosely over the top of the Passport. Do **not** place a dressing between the external bolster and the skin.

12. Upon completion of the procedure, dispose of the contents of the kit per institutional guidelines for biohazardous medical waste.

Caution: Items required for delivery of nutrition and medication as well as for gastric drainage or decompression are contained in the Patient Care Kit included with this set. This kit **must** accompany the patient home or to the care facility.

PASSPORT REMOVAL

1. To verify there is no tissue adherence to the tube, hold the Passport near the stoma. Slowly rotate the tube and gently push the external bumper. **Warning:** If the tube does not move freely when performing these steps, **do not proceed** with external/traction removal. Endoscopic assessment may be necessary.
2. Place the fingers of one hand flat against the skin surrounding the Passport to apply gentle, constant, counter pressure while removing the Passport. Holding the Passport external bolster straight, apply steady

traction until the conical tip emerges through the stoma. **Caution: The Passport must be retracted perpendicular to the skin, in alignment with the stoma tract. Warning: Do not insert anything into the Passport during the removal procedure as damage to the stoma tract may result.**

3. The size of the replacement Passport may be determined by referencing the size marked on the rim of the existing Passport or by using the Catheter Measuring Device.

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Passport is a trademark of Wilson-Cook Medical Inc.
Cook is a registered trademark of Cook Incorporated.
Wilson-Cook Medical Inc.
Cook Ireland Ltd.