Disposable Varices Injectors

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18886/0105

COOK[®] Wilson-Cook Medical
GI Endoscopy

English	1
Português	3
Español	6

INTENDED USE

The Wilson-Cook Disposable Varices Injectors are used for endoscopic injection into gastrointestinal mucosa. The Disposable Varices Injectors are available with a Luer handle. These devices are supplied sterile and are intended for single use only.

NOTES

Do not use this device for any purpose other than the stated intended use.

Inventory rotation of sterile products is essential. Verify the expiration date on the package label prior to using the product. If the expiration date has lapsed, do not use or resterilize the device.

If the product package is open or damaged when received, do not use this device.

Wilson-Cook devices must be stored in a dry location, away from temperature extremes.

CONTRAINDICATIONS

Contraindications include those specific to the primary endoscopic procedure to be performed in gaining access to the desired injection site.

Severe coagulopathy.

POTENTIAL COMPLICATIONS

Potential complications associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Potential complications associated with mucosal injection include, but are not limited to: ulceration or necrosis of injected tissue, venous thrombosis, stricture formation.

Other potential complications to be considered if injection is performed in the esophagus include, but are not limited to: chest pain, mediastinitis, pleural effusion.

PRECAUTION

The coordination of the endoscope accessory channel size with compatible devices is essential in obtaining optimal results during a procedure. Please refer to the product package label for the minimum channel size required for this device.

CAUTIONS

A thorough understanding of the technical principles, clinical applications and risks associated with GI endoscopy and mucosal injection is necessary before using this device. Disposable Varices Injectors should

only be used by, or under the supervision of, physicians thoroughly trained in therapeutic endoscopy and mucosal injection.

The needle must be retracted into the sheath prior to introduction, advancement, or withdrawal of the device. Failure to retract the needle may result in damage to the endoscope.

PRODUCT INSPECTION

1. Upon removing the device from the package, visually inspect the device with particular attention to kinks, bends, or breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook for return authorization.
2. Uncoil the catheter and straighten completely.
3. Ensure needle advances and retracts by pushing forward then back on the inner catheter.

INSTRUCTIONS FOR USE

1. With the needle completely advanced, attach a pre-filled syringe to the Luer lock fitting on the device, then flush air from the needle. Check for patency and leaks. **Caution:** Prior to introducing the device into the accessory channel of the endoscope, retract the needle into the sheath to avoid puncturing the channel wall of the endoscope.
2. Ensure the needle is fully retracted to just inside the outer catheter and introduce the device into the accessory channel of the endoscope.
3. Advance the device in short increments until it is visualized exiting the endoscope.
4. Endoscopically visualize the desired injection site and maintain the position of the endoscope.
5. Advance the needle into the mucosa, then using the attached syringe, perform injection. **Caution:** During needle advancement, ensure the sheath of the device has exited the endoscope completely and is within endoscopic view.
6. When injection is complete, withdraw the needle from the injection site and retract it into the sheath. Repeat the procedure as necessary.
7. Upon completion of the procedure, retract the needle into the sheath, then remove the device from the endoscope.
8. Dispose of the device per institutional guidelines for biohazardous medical waste.

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Cook is a registered trademark of Cook Incorporated.
Wilson-Cook Medical Inc.
Cook Ireland Ltd.