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INTENDED USE

This device is used for endoscopic clip placement within the gastrointestinal tract for the purpose of endoscopic marking, hemostasis for mucosal/submucosal defects less than 3 cm in the upper GI tract, bleeding ulcers, arteries less than 2 mm, and polyps less than 1.5 cm in diameter in the GI tract. This device is not intended for the repair of GI tract luminal perforations. The device is supplied sterile and is intended for single use only.

NOTES

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook for return authorization.

The number of clips required for hemostasis may vary depending upon the anatomical site, histology, lesion type, and patient condition and history. A sufficient quantity of clips should be prepared in consideration of all of these factors prior to the procedure.

CONTRAINDICATIONS

Contraindications include those specific to the primary endoscopic procedure to be performed in gaining access to the desired site.

Contraindications to endoscopic hemostasis include, but are not limited to: uncooperative patient, coagulopathy, cricopharyngeal or esophageal narrowing or stricture, and tortuous esophagus.

POTENTIAL COMPLICATIONS

Potential complications associated with gastrointestinal endoscopy and endoscopic hemostasis include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest, hematemesis, transient dysphasia, aspiration pneumonia, wound dehiscence, minimal acute inflammatory tissue reaction, transitory local irritation, migration of clip into the bile duct, and anatomy disruption.

PRECAUTIONS

Refer to the package label for the minimum channel size required for this device.

The endoscope must remain as straight as possible when inserting or withdrawing the device.

Endoscopic hemostasis should be performed under direct endoscopic visualization.

Clipping hard or severely fibrotic lesions to achieve hemostasis may be more difficult.

If the clip deployment device is used with the endoscope in a torqued or retroflexed position, clip deployment difficulties can occur.

Lesions located in the esophagus and the lesser curvature of the stomach may be difficult to treat with a forward-viewing endoscope.

The treatment of esophageal varices may require clipping in combination with a sclerosing agent.

The use of clips in the presence of bacterial contamination may potentiate or prolong infection.

WARNINGS

Failure to deploy clip or improper deployment of a clip may require surgical intervention. Wire cutters may be used to cut the wire and sheath.

Electrosurgical devices used after clipping may cause patient injury.

MRI compatibility of TriClip endoscopic clips has not been established.

SYSTEM PREPARATION

Note: See fig. 1 prior to use for component identification. **Caution:** Do not remove clip release tab until clip is in the desired position to grasp tissue; this may cause premature clip deployment and/or patient injury.

1. Ensure that a clip is securely attached to the introducer by pulling gently on the clip. Also, confirm that the three prongs of the clip are spread apart. If the clip is not attached, align the notch at the distal end of the cable with the corresponding notch at the base of a clip.

Note: Ensure the spool is in its original position. Then, slide the metal sleeve of the clip over the intersecting notches. (See fig. 2) Confirm that the clip is attached by pulling gently on the clip. **Note:** If clip release tab was removed, replace it.

2. Unscrew luer lock and slowly advance the sheath extender forward until the clip is completely retracted within the outer sheath. **Note:** To facilitate smooth retraction, guide the clip into the outer sheath while advancing the sheath extender. **Caution:** If the clip is not completely retracted within the outer sheath, injury to the patient, the device and/or the endoscope may occur. **Note:** If present, remove flared tip after retraction.

INSTRUCTIONS FOR USE

1. After visually determining the appropriate clipping site, advance the device into the endoscope accessory channel in small increments.
2. Endoscopically confirm that the device has exited the endoscope.
3. Position the distal tip of the device toward the targeted site.
4. To expose the clip, pull the sheath extender towards the handle. Using the Luer lock, connect the sheath extender to the handle. (See fig. 3)

Note: The clip cannot be retracted into the outer sheath while the device is in the endoscope.

5. Position clip at targeted tissue site. **Note:** If applicable, flush as desired.
6. When satisfied with the position of the clip, remove the clip release tab. (See fig. 4)
7. Grasp the stem grip with one hand and grasp the clip spool with the other hand. Advance the clip spool toward the stem grip until the prongs of the clip begin to compress onto the tissue. **Note:** The farther the clip spool is extended, the tighter the clip will grasp onto the tissue. **Note:** Keep clip pressed firmly against target site for maximum tissue capture. (See fig. 5)
8. When the clip is positioned on the targeted site, release the clip from the device by slowly pulling the clip spool toward the backstop. (See fig. 6) **Caution:** Failure to completely release the clip may cause the clip to pull away from the closure site.
9. After deployment, advance sheath extender forward and remove the device from the endoscope.
10. Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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© 2005 Wilson-Cook Medical Inc.
TriClip is a trademark of Wilson-Cook Medical Inc.
Cook is a registered trademark of Cook Incorporated.
Wilson-Cook Medical Inc.
Cook Ireland Ltd.