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18906/0105

COOK[®] Wilson-Cook Medical
GI Endoscopy

English	1
Português/Portuguese	4
Español/Spanish	7

INTENDED USE

The Colon Decompression Set is used for treatment of acute non-toxic megacolon, pseudo-obstruction (Ogilvie's syndrome) and colonic strictures. The Colon Decompression Set includes a drainage catheter, guiding catheter and .035 inch wire guide. The Colon Decompression Set is supplied sterile and is intended for single use only.

NOTES

Do not use this device for any purpose other than the stated intended use.

Inventory rotation of sterile product is essential. Verify the expiration date on the package label of sterile devices prior to using the product. If the expiration date has lapsed, do not use or resterilize the device.

If the package is open or damaged when received, do not use the device.

Wilson-Cook devices must be stored in a dry location, away from temperature extremes.

CONTRAINDICATIONS

Contraindications of colonoscopy include, but are not limited to: fulminant colitis, possible existing perforation, acute severe diverticulitis, presence of barium or inadequate bowel preparation.

Contraindications of colon decompression include, but are not limited to: ischemic bowel, recent rectal surgery, organic colon obstruction, recent surgical anastomosis, acute myocardial infarction.

POTENTIAL COMPLICATIONS

Potential complications associated with colonoscopy include, but are not limited to: perforation, hemorrhage, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Other potential complications associated with colon decompression include, but are not limited to: clogging of the tube, dislodgment of the tube as colonoscope is withdrawn, perforation.

PRECAUTIONS

The coordination of the endoscope accessory channel size with compatible devices is essential in obtaining optimal results during a procedure. The Colon Decompression Set requires a minimum working channel of 2.8 mm.

Careful history and physical examination must be performed prior to tube insertion in order to determine the underlying cause of the pseudo-obstruction.

The colon should not be prepped unless there is an inordinate amount of stool in the rectum.

CAUTIONS

A thorough understanding of the technical principles, clinical applications and risks associated with colonoscopy and colon decompression is necessary before using this device. The Colon Decompression Set should only be used by, or under the supervision of, physicians thoroughly trained in therapeutic colonoscopy and colon decompression.

Wilson-Cook wire guides are recommended for use with this device.

INSTRUCTIONS FOR USE

1. Upon removing the device from the package, visually inspect with particular attention to kinks, bends, or breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook Medical for return authorization.
2. Advance the colonoscope, insufflating minimal air, until it has reached the cecum or as far in the colon as possible. **Note:** Suction must be performed intermittently throughout the procedure to help decompress the colon and avoid further colonic distention.
3. Fluoroscopically observe the location of the colonoscope tip.
4. Insert the .035 inch wire guide into the biopsy channel of the colonoscope until it exits the tip.
5. Carefully withdraw the colonoscope, making certain the wire guide remains stationary within the colon by periodic fluoroscopic monitoring.
6. Disassemble the colon decompression tube and pre-loaded 6 FR guiding catheter.
7. Flush the colon decompression tube and 6 FR guiding catheter with sterile water or lubricate with water-soluble lubricant, then reassemble and advance into the colon over the pre-positioned wire guide. Use fluoroscopic control to avoid migration of the wire guide.
Caution: Difficulty inserting the decompression tube may occur due to the formation of a sigmoid loop. This may be reduced by carefully withdrawing the decompression tube and guiding catheter/wire guide assembly a short distance, while observing loop reduction fluoroscopically.
8. Once the tube is advanced to the end of the wire guide, withdraw both the guiding catheter and wire guide, leaving the decompression tube in place.
9. Secure the external portion of the tube to the sacral crease by taping a gauze sponge to pad the area.
10. Attach the tube to a low intermittent suction apparatus and irrigate with 15 ml of water approximately every 2 hours in order to reduce clogging of the tube aspiration ports.

- 
11. Withdraw the tube as colonic function returns, then dispose of the device per institutional guidelines for biohazardous medical waste.
Note: The colon may gradually expel the tube with the return of motility.

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© 2005 Wilson-Cook Medical Inc.
Cook is a registered trademark of Cook Incorporated.
Wilson-Cook Medical Inc.
Cook Ireland Ltd.