**Cytology Brushes**

**Escovas de Citologia**

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18910/0205

**COOK**<sup>®</sup> Wilson-Cook Medical  
GI Endoscopy

English . . . . .	1
Português/Portuguese . . . . .	3
Español/Spanish . . . . .	5

## **INTENDED USE**

The Wilson-Cook Cytology Brushes are used to collect cells from the bronchi and upper and lower gastrointestinal tracts. This device is supplied sterile and intended for single use only.

## **NOTES**

Do not use this device for any purpose other than the stated intended use.

Inventory rotation of sterile products is essential. Verify the expiration date on the package label prior to using the product. If the expiration date has lapsed, do not use or resterilize the device.

If the product package is open or damaged when received, do not use this device.

Wilson-Cook devices must be stored in a dry location, away from temperature extremes.

## **CONTRAINDICATIONS**

Contraindications include those specific to the primary endoscopic procedure to be performed in gaining access to the desired cytology site.

## **POTENTIAL COMPLICATIONS**

Potential complications associated with pulmonary or gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

## **PRECAUTION**

The coordination of the endoscope accessory channel size with compatible devices is essential in order to obtain optimal results during a procedure. Cytology brushes are either 5.0 French requiring a minimum channel size of 2.0 mm, or 7.0 French requiring a minimum channel size of 2.8 mm.

## **CAUTION**

A thorough understanding of the technical principles, clinical applications and risks associated with pulmonary or gastrointestinal endoscopy and cytology is necessary before using this device. Wilson-Cook Cytology Brushes should only be used by, or under the supervision of physicians thoroughly trained in pulmonary or gastrointestinal endoscopy and cytology.

## **INSTRUCTIONS FOR USE**

1. Upon removal of the device from the package, visually inspect with particular attention to kinks, bends or breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook for return authorization.

2. Endoscopically inspect the area to be sampled.
3. With the cytology brush retracted, insert the device into the accessory channel of the endoscope.
4. Advance in 1-2 cm increments until the tip of the device is visualized exiting the endoscope.
5. Advance the device to the desired sampling site, then advance the brush out of the sheath by pushing forward on the handle.
6. Brush the area to be sampled several times to obtain adequate cellular material.
7. Pull the brush back into the sheath and withdraw the device from the endoscope.
8. Prepare the specimen for examination per institutional guidelines.
9. Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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© 2005 Wilson-Cook Medical Inc.
Cook is a registered trademark of Cook Incorporated.
**Wilson-Cook Medical Inc.**
**Cook Ireland Ltd.**