**COOK®**

**MEDICAL**

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## INTENDED USE

This device is used for soft-tissue biopsy through the accessory channel of an ultrasound endoscope.

## NOTES

Do not use this device for any purpose other than stated intended use.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Store in a dry location, away from temperature extremes.

## CONTRAINDICATIONS

Those specific to the primary endoscopic procedure to be performed in gaining access to desired biopsy site.

Coagulopathy.

## POTENTIAL COMPLICATIONS

Those associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

## PRECAUTIONS

Refer to package label for minimum channel size required for this device.

Needle must be retracted into sheath and thumbscrew ring locked at zero centimeter mark prior to withdrawal of device. Failure to retract needle may result in damage to ultrasound endoscope.

## INSTRUCTIONS FOR USE

1. Prior to procedure, advance device into accessory channel to ensure sufficient device length. **Note:** If device length is insufficient to perform procedure, adjustable spacer may be removed. (See fig. 1)

**Note:** Proper needle orientation is essential to obtain optimal tissue samples. (See fig. 2)

2. Identify desired biopsy site based on ultrasound imaging.
3. Prepare needle by pulling back on plunger until a firm click is felt indicating that needle is locked in position then position needle flush with distal end of catheter and lock in place with thumbscrew ring. Introduce needle into accessory channel.
4. Advance device in small increments until Luer lock fitting at base of spacer meets fitting on accessory channel. **Note:** If resistance is encountered during passage of device through accessory channel, retract and straighten ultrasound endoscope and reattempt passage.

5. Attach device to ultrasound endoscope accessory channel port by rotating device handle until fittings are connected. **Caution:** Ensure device has been attached to accessory channel. Failure to attach device prior to and during needle adjustment or extension may result in damage to ultrasound endoscope.
6. To adjust catheter extension from ultrasound endoscope, disengage handle at adjustment wheel (*See fig. 1*) and rotate adjustment wheel until proper catheter extension from ultrasound endoscope is obtained. Reattach handle to adjustment wheel.
7. While maintaining ultrasound endoscope position, adjust needle extension to desired length by loosening thumbscrew on handle, setting ring to desired extension, then tightening thumbscrew.
8. Using handle, advance prepared needle into targeted tissue. **Warning:** As plunger is advanced, stylet with specimen notch will extend 2 cm beyond distal tip of needle. Therefore, to prevent stylet extension beyond biopsy site, needle must be retracted 2 cm prior to advancing stylet. **Caution:** Rigidity of device may prohibit access to duodenum. **Note:** Quick-Core Needle allows endosonography visualization during ultrasound.
9. To expose specimen notch for biopsy, advance stylet from needle by gently pressing on plunger until resistance is felt. **Caution:** Once resistance is felt, stop pressing plunger. Continuing to press plunger will cause needle to inadvertently advance over stylet.
10. Reconfirm proper needle position.
11. To biopsy, fully depress plunger, applying moderate force in direction of accessory channel. This action will cause needle to rapidly advance, capturing tissue.
12. Upon completion of tissue collection, completely retract needle into sheath by pulling back on handle shaft and lock thumbscrew ring.
13. Disconnect Luer lock fitting of device from accessory channel and withdraw entire device from ultrasound endoscope.
14. Unlock thumbscrew ring and extend needle. To remove tissue specimen, pull back on plunger until a firm click is felt indicating that needle is locked in position. To expose tissue specimen within notch, gently push stylet forward until resistance is felt. **Caution:** Once resistance is felt, discontinue advancing stylet. Failure to do so will cause needle to inadvertently rapidly advance.
15. Remove tissue from specimen notch.
16. For additional tissue collection from same lesion, return needle to its original position by depressing plunger with thumb.
17. Additional samples may be obtained by repeating Steps 2-16.

**Upon completion of procedure, dispose of device per institutional guidelines for biohazardous medical waste.**

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Quick-Core and Cook are registered trademarks of Cook Incorporated.
**Wilson-Cook Medical, Inc.**
**Cook Ireland Ltd.**