**ShortShot™ Saeed Hemorrhoidal
Multi-Band Ligator**

**Ligador de Multi-Bandas
de Hemorróides ShortShot™ Saeed**

**Sistema de ligadura hemorroidal
multibanda ShortShot™ Saeed**

18919/0205

COOK® Wilson-Cook Medical
GI Endoscopy

English	1
Português/Portuguese	4
Español/Spanish	7

INTENDED USE

The ShortShot Saeed Hemorrhoidal Multi-Band Ligator is used to ligate internal hemorrhoids. This device is supplied non-sterile and is intended for single use only.

NOTES

Do not use this device for any purpose other than the stated intended use.

If the package is open or damaged when received, do not use the device.

Inventory rotation of this device is essential. Verify the expiration date on the package label. If the expiration date has lapsed, do not use the device.

The device should be stored in a dry location, away from temperature extremes.

CONTRAINDICATIONS

Contraindications include those specific to the primary anoscopic procedure to be performed in gaining access to the desired banding site.

Contraindications specific to hemorrhoidal banding include, but are not limited to: coagulopathy, anal stricture, and active inflammatory bowel disease in the distal rectum.

Use of ligation bands is contraindicated in patients with a known hypersensitivity to latex.

POTENTIAL COMPLICATIONS

Potential complications associated with anoscopy include, but are not limited to: perforation and hemorrhage.

Potential complications associated with sedation include, but are not limited to: allergic reaction to medication, hypotension, cardiac arrhythmia or arrest, respiratory depression or arrest.

Potential complications associated with hemorrhoidal banding include, but are not limited to: hemorrhage, fever, infection, nausea, urinary retention, stricture formation, and obstruction.

PRECAUTIONS

The inner lumen diameter of the anoscope must be compatible with the ShortShot to obtain optimal results during a procedure. Ensure the ShortShot passes through the channel of the preferred scope with adequate clearance to avoid dislodging bands.

Ligation should only be performed on hemorrhoids located above the dentate line; treatment below the dentate line will result in pain.

CAUTIONS

A thorough understanding of the technical principles, clinical applications and risks associated with anoscopy and hemorrhoidal banding is necessary

before using this device. The ShortShot should only be used by, or under the supervision of, physicians thoroughly trained in anoscopy and band ligation.

Banding should begin at the most proximal location from the anal sphincter and proceed distally because passing the anoscope over a previously placed band may dislodge the band.

Prior to using the ShortShot, a routine anoscopic examination is recommended to confirm the diagnosis requiring treatment of hemorrhoids.

COMPONENTS

ShortShot Saeed Hemorrhoidal Multi-Band Ligator with preloaded bands.

EQUIPMENT NEEDED

- Appropriate size anoscope
- External suction source
- Water-soluble lubricant

SYSTEM PREPARATION

1. Upon removal of the ShortShot from the package, visually inspect the device with particular attention to joints, cracks, kinks, or breaks. If a defect is detected, do not use. Please notify Wilson-Cook for return authorization.
2. The ligator barrel should maneuver easily within the anoscope channel.
Note: Prior to procedure, test maneuverability of ligator barrel within the channel of the anoscope.
3. Connect suction tube to graduated connector at bottom of ligator handle. Place finger on the suction port and verify suction is present at the barrel tip. The ShortShot is now ready for ligation of hemorrhoids.

INSTRUCTIONS FOR USE

Note: An anoscopic or screening exam of the hemorrhoidal site should be performed prior to use of the ShortShot.

1. Lubricate and introduce the anoscope.
2. Position the anoscope to visualize the hemorrhoid selected for ligation. (See fig. 1) Insert the ShortShot through the anoscope. **Caution:** Be careful not to cover suction port prior to achieving desired ligation position.
3. After full contact is made between the barrel tip of the ligator and the hemorrhoid, suction the hemorrhoid into the ligator by covering the suction port.
4. Maintain suction of the internal hemorrhoid and deploy the band by slowly rotating spool downward until tension is released. (See fig. 2)
Note: Maintain suction during band deployment by keeping suction port covered.

5. Uncover suction port of ligator to relieve suction and slowly remove ligator from the ligated hemorrhoid.
6. Repeat ligation process as needed. If additional bands are required, attach suction tubing to new ShortShot.
7. Upon completion of the ligation procedure(s), remove the ShortShot and anoscope from the patient.
8. Dispose of the devices per institutional guidelines for biohazardous medical waste.

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ShortShot is a trademark of Wilson-Cook Medical Inc.
Cook is a registered trademark of Cook Incorporated.
Wilson-Cook Medical Inc.
Cook Ireland Ltd.