**Biliary Wire Guide**

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**Galdevejskateterleder**

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**Guide biliaire**

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## INTENDED USE

This device is used to assist in cannulation of the biliary and pancreatic ducts and to aid in bridging difficult strictures during ERCP. Placement of the wire allows for continual access to the desired duct and facilitates exchange of compatible wire-guided accessories. This device is supplied sterile and intended for single use only.

## NOTES

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Wilson-Cook for return authorization.

## CONTRAINDICATIONS

Contraindications include those specific to ERCP.

## POTENTIAL ADVERSE EVENTS

Potential adverse events associated with ERCP include, but are not limited to: perforation, allergic reaction to contrast or medication.

## PRECAUTIONS

Refer to package label for minimum channel size required for this device.

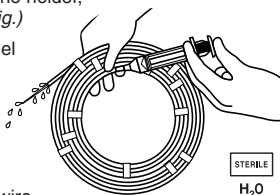
Use of this wire guide with metal tip ERCP devices may result in damage to the external coating and/or tip of the wire guide.

## WARNING

Remove this wire guide before applying electrical current.

## INSTRUCTIONS FOR USE

1. Prior to removing the wire guide from the holder, flush with 30 cc of sterile water. (*See fig.*)
2. Flush the endoscope accessory channel and/or the lumen of the device with sterile water, then insert the wire guide.
3. Fluoroscopically monitor wire guide advancement in the ductal system.
4. Proceed with exchange of compatible wire-guided accessories over the wire guide.
5. Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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Cook is a registered trademark of Cook Incorporated.
**Wilson-Cook Medical Inc.**
**Cook Ireland Ltd.**