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## INTENDED USE

This device is used for the endoscopic cannulation of the ductal system and for removal of biliary stones. This device is supplied sterile and intended for single use only.

## NOTES

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook Endoscopy for return authorization.

## CONTRAINDICATIONS

Contraindications include those specific to ERCP and any procedures to be performed in conjunction with balloon stone extraction.

Use of this natural latex rubber balloon is contraindicated in patients with a known hypersensitivity to latex.

## POTENTIAL COMPLICATIONS

Potential complications associated with ERCP include, but are not limited to: pancreatitis, cholangitis, aspiration, perforation, hemorrhage, infection, sepsis, allergic reaction to contrast or medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Additional complications that can occur during endoscopic balloon stone extraction include, but are not limited to: stone impaction, localized inflammation, pressure necrosis.

## PRECAUTIONS

Refer to the package label for the minimum channel size required for this device.

Assessment of the size of the stone and ampullary orifice must be made to determine the necessity of sphincterotomy. In the event sphincterotomy is required, all necessary precautions must be followed.

Cook Endoscopy wire guides are recommended for use with this product.

Injection of contrast during ERCP should be monitored fluoroscopically. Overfilling of the pancreatic duct may cause pancreatitis.

## INSTRUCTIONS FOR USE

1. Ensure that all hubs are secure. Verify balloon integrity prior to use by opening the stopcock that is attached to the **balloon inflation port**, attaching the enclosed **premeasured syringe** to the stopcock, then inflating the balloon with **AIR ONLY**. If any leakage is detected, do not use. **Note:** The stopcock is in the open position, allowing access to the balloon, when the two arms are aligned with the catheter and syringe. To maintain balloon inflation, turn the stopcock arm 90°.

2. With the elevator open, advance the deflated balloon over a pre-positioned wire guide, through the accessory channel in short increments until it is visualized exiting the endoscope. **Note:** Prior to advancing the device over a pre-positioned wire guide, close the Tuohy-Borst adapter and flush the wire guide lumen with sterile water to facilitate advancement.
3. Position the deflated balloon. **Note:** Two radiopaque bands, one on either side of the balloon, provide fluoroscopic visualization of the balloon position.
4. Inflate balloon. Maintain inflation by turning balloon port stopcock 90° to the closed position.
5. Advance .025 inch wire guide through the blue port. **Note:** The floppy tip of the .025 inch wire guide may deflect off the inflated balloon into the desired position. **Note:** If injecting contrast, close Tuohy-Borst adapter and attach syringe to extension tube after removing protective cap.
6. If necessary, stone extraction may now be performed. **Note:** If more than one stone is to be removed, extract one stone at a time.
7. Keeping the elevator open, gently withdraw the inflated balloon and stone toward the papilla under fluoroscopic monitoring. A black 1 cm band precedes the balloon by 2 cm allowing endoscopic awareness of the balloon approaching the papilla. **Warning:** Do not exert excessive pressure on the ampulla while extracting stones. If the stone does not pass easily, reassess the need for sphincterotomy.
8. Repeat the extraction process until the duct is clear.
9. Once the balloon is visualized in the duodenum, turn the stopcock 90° to the open position and deflate the balloon.
10. Withdraw the deflated balloon from the accessory channel. **Note:** When removing the balloon from the endoscope, the wire guides may be left in place to facilitate the introduction of other compatible Cook Endoscopy wire-guided devices.
11. Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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Bouncer is a trademark of Wilson-Cook Medical Inc.
Cook is a registered trademark of Cook Incorporated.
**Cook Ireland Ltd.**