Biliary Manometry Catheters

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Katetre til galdemanometri

Biliaire manometriekatheters

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Gallengang-Manometriekatheter

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INTENDED USE

This device is used to obtain pressure measurements from the Sphincter of Oddi and common bile duct. This wire-guided catheter is supplied sterile and intended for single use only.

NOTES

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook Endoscopy for return authorization.

CONTRAINDICATIONS

Contraindications include those specific to ERCP.

POTENTIAL COMPLICATIONS

Potential complications associated with ERCP and Sphincter of Oddi manometry include, but are not limited to: pancreatitis, cholangitis, sepsis, perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction to contrast or medication, respiratory depression or arrest, cardiac arrhythmia or arrest.

PRECAUTIONS

Refer to the package label for the minimum channel size required for this device.

Cook Endoscopy wire guides are recommended for use with this device.

INSTRUCTIONS FOR USE

1. Attach each perfusion lumen to the transducers and fill with sterile water. **Note:** Set up and standardization may vary. Follow recording and perfusion system manufacturer's instructions. **Note:** The catheter is equipped with three color coded perfusion lumens. The red lumen is the proximal port, the yellow lumen is the middle port and the blue lumen is the distal port.
2. Insert the manometry catheter into the endoscope accessory channel, over the pre-positioned wire guide, and proceed with cannulation of the selected ductal system.
3. Advance the catheter into the desired position, remove the wire guide, and proceed with Sphincter of Oddi manometry per institutional guidelines. **Note:** There are ten 1 mm black markings on the distal end of the catheter to aid in visualization. The three perfusion holes are 2 mm apart and begin at the distal end of the black markings.
4. Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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Cook is a registered trademark of Cook Incorporated.
Cook Ireland Ltd.