Memory™ Baskets 5FR

Memory™ 5 FR 网篮

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Memory™ kurve 5 Fr

Memory™ mandjes 5 Fr.

Paniers Memory™ 5 Fr.

Memory™ Körbchen, 5 Fr.

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INTENDED USE

This device is used for the endoscopic removal of biliary stones and foreign bodies. This device is supplied sterile and intended for single use only.

NOTE

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook Endoscopy for return authorization.

Store in a dry location, away from temperature extremes.

CONTRAINDICATIONS

Contraindications include those specific to the primary endoscopic procedure to be performed in conjunction with the removal of the foreign body.

Perforation.

If the Memory 5FR Basket is to be used for biliary stone extraction, contraindications include an ampullary opening inadequate to allow for the unimpeded passage of the stone and basket.

POTENTIAL COMPLICATIONS

Potential complications associated with gastrointestinal endoscopy and ERCP include, but are not limited to: pancreatitis, cholangitis, sepsis, perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction, respiratory depression or arrest, cardiac arrhythmia or arrest.

Other potential complications associated with basket extraction include, but are not limited to: impaction of the object, aspiration of foreign body, localized inflammation, pressure necrosis.

PRECAUTIONS

Refer to the package label for the minimum channel size required for this device.

If this device is to be used for the removal of biliary stones, assessment of the stone size and ampullary orifice must be made to determine the necessity of sphincterotomy. In the event sphincterotomy is required, all appropriate cautions, warnings and contraindications must be observed.

Injection of contrast during ERCP should be monitored fluoroscopically. Overfilling of the pancreatic duct may cause pancreatitis.

Memory 5FR Baskets should never be coiled in less than an 8-inch (20 cm) diameter.

WARNINGS

Cook Endoscopy Memory 5FR Baskets are **not** compatible with the Soehendra[®] Lithotripter or any other mechanical lithotripter.

When retrieving a foreign body from the upper GI tract, extreme caution must be exercised to avoid occlusion of the airway.

Surgical intervention may be required if impaction occurs.

INSTRUCTIONS FOR USE

1. With the basket fully retracted into the sheath, insert the device into the endoscope accessory channel. Advance the device through the channel, in short increments, until the basket sheath exits the endoscope.
2. Endoscopically or fluoroscopically visualize the extraction target and advance the basket to the appropriate position. Fluoroscopic visualization may be enhanced by injecting contrast via the side arm fitting. Place the sheath of the basket beyond the target before extending the basket to entrap the extraction target.
3. Confirm the desired position of the basket sheath relative to the target. With one hand holding the side arm fitting, **gently** push forward on the pin vise handle with the other hand to advance the basket out of the sheath. **Caution:** Pulling on the sheath while advancing or retracting the basket may damage the device, rendering it inoperable.
4. Place the basket around the extraction target or foreign body and slowly retract the handle until entrapment is achieved. **Note:** It may be necessary to move the basket back and forth along the target in order to capture the object.
5. Withdraw the basket into the channel until the foreign body or biliary stone is supported against the endoscope tip, then withdraw the endoscope from the patient. Maintain endoscopic visualization to ensure continued entrapment of the object. While withdrawing the device from the endoscope, wipe the secretions from the outer sheath. **Caution:** If difficulty is encountered when removing the basket from the duct, moderate force may be applied by pulling on the pin vise handle to manually fracture the stone. If passage is still restricted, surgical intervention may be necessary.
6. Prepare or dispose of the extracted specimen per institutional guidelines.
7. Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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Memory is a trademark of Cook Endoscopy.
Soehendra is a registered trademark of Cook Endoscopy.
Cook is a registered trademark of Cook Incorporated.
Cook Ireland Ltd.