# Surgisis® Biodesign™

DURAL GRAFT

FP0030-1F

## Manufacturer

Cook Biotech Inc.  
1425 Innovation Place  
West Lafayette, IN 47906, U.S.A.  
TEL: 812-339-2235  
Toll free: 800-457-4500  
FAX: 800-554-8335

## EU Representative

Cook Ireland  
O'Halloran Road  
National Technological Park  
Limerick, IRELAND  
TEL: 353-613-34440  
FAX: 353-613-34441

© Cook Biotech Incorporated 2007

## ENGLISH

**DESCRIPTION:** Cook® Surgisis® Biodesign™ Dural Graft is a nonporous, absorbable multi-layer sheet of extracellular collagen matrix derived from porcine small intestinal submucosa. The Dural Graft is non-pyrogenic and has sufficiently low endotoxin levels to make it suitable for applications where it will contact cerebrospinal fluid.

**INTENDED USE:** The Dural Graft is intended for use as a dural substitute for repairing dura mater. This device is supplied sterile in peel-open packages and is intended for one-time use.

**Rx ONLY** This symbol means the following:

**CAUTION:** Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

This device is intended for use by trained medical professionals.

**CONTRAINDICATIONS:** This device should not be implanted into areas with known or suspected infection or in necrotic tissue. This device is derived from a porcine source and should not be used for patients with known sensitivity to porcine material.

## PRECAUTIONS:

- **Do not resterilize.** Do not subject this device to steam, gas (ethylene oxide), or radiation sterilization as these may damage the device.
- Device performance when placed in tissue having compromised circulation has not been established.
- Discard all open and unused portions of the Dural Graft sheets.
- Device is sterile and non-pyrogenic if the package is dry, unopened, and undamaged.
- Discard device if damage or contamination is observed, or if the device is past its expiration date.
- Prior to touching the device, wash surgical gloves thoroughly to remove the glove powder.
- Ensure that device is rehydrated before suturing.
- Ensure that all layers of the Dural Graft are secured when suturing.

**POTENTIAL COMPLICATIONS:** The following complications are possible with the use of surgical graft materials.

- Infection, adhesion, CSF leak, and calcification
- Acute or chronic inflammation (Initial application of surgical graft materials may be associated with transient, mild, localized inflammation.)
- Allergic reaction

**STORAGE:** This device should be stored in a clean, dry location at controlled room temperature. Do not freeze this device.

**STERILIZATION:** This device has been sterilized with ethylene oxide.

## SUGGESTED INSTRUCTIONS FOR USE

These recommendations are designed to serve only as a general guideline. They are not intended to supersede institutional protocols or professional clinical judgment concerning patient care.

**NOTE: Always handle the Dural Graft sheets using aseptic technique. Minimize contact with latex gloves.**

## Required Materials

- A sterile dish (kidney dish or other bowl)
  - Sterile forceps
  - Rehydration fluid: at least 100 mL of sterile, room temperature, saline or sterile, room temperature lactated Ringer's solution for each Dural Graft sheet.
1. Using aseptic technique, remove the Dural Graft inner pouch from its outer bag, and place the inner pouch in the sterile field.
  2. Open the inner pouch carefully, and aseptically remove the Dural Graft sheet with the sterile forceps.
  3. Place the Dural Graft sheet into the sterile dish in the sterile field. (Multiple Dural Graft sheets may be rehydrated simultaneously in the same dish.)
  4. Add to the dish at least 100 mL of the rehydration fluid for each Dural Graft sheet.
  5. Allow Dural Graft sheets to rehydrate for at least five minutes.
  6. Using aseptic technique, trim the Dural Graft sheet to fit the site, providing a small allowance for overlap. (Note: An alternative method is to cut the Dural Graft sheet to size prior to rehydration. If this method is selected, be sure to rehydrate the Dural Graft sheet prior to suturing it into place. See step 5.)
  7. Using aseptic technique, transfer the Dural Graft sheet to the graft site and suture into place, avoiding excess tension.  
**Note:** Surgical experience indicates that suturing Dural Graft sheets with close tissue approximation produces better outcomes.
  8. Complete the standard surgical procedure.
  9. Discard any unused portions of the Dural Graft sheet following standard technique for disposal of medical waste.

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