|                                     |                 |
|-------------------------------------|-----------------|
| <b>Cook® Fistula Brush</b>          | <b>EN</b><br>3  |
| <b>Píštělový kartáček Cook®</b>     | <b>CS</b><br>3  |
| <b>Cook® fistelbørste</b>           | <b>DA</b><br>4  |
| <b>Cook® Fistelbürste</b>           | <b>DE</b><br>5  |
| <b>Βούρτσα συριγγίου Cook®</b>      | <b>EL</b><br>5  |
| <b>Cepillo para fístulas Cook®</b>  | <b>ES</b><br>6  |
| <b>Brosse pour fistule Cook®</b>    | <b>FR</b><br>7  |
| <b>Cook® sipolykefe</b>             | <b>HU</b><br>7  |
| <b>Spazzolino per fistola Cook®</b> | <b>IT</b><br>8  |
| <b>Cook® fistelborstel</b>          | <b>NL</b><br>9  |
| <b>Cook® fistelbørste</b>           | <b>NO</b><br>9  |
| <b>Szczoteczka do przetok Cook®</b> | <b>PL</b><br>10 |
| <b>Escova para fístula Cook®</b>    | <b>PT</b><br>11 |
| <b>Cook® fistelborste</b>           | <b>SV</b><br>11 |
| <b>Cook® 瘻管刷</b>                    | <b>ZH</b><br>12 |

U \_ C F B \_ R E V . 1

## COOK® FISTULA BRUSH

Used to identify, clean, or debride a rectal fistula tract, and facilitate placement of a Surgisis® Biodesign™ Recto-Vaginal/Anal Fistula Plug.

**CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.**

**CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.**

### CONTRAINDICATIONS

Not for vascular use.

### CAUTIONS

- Supplied sterile in peel-open package.
- Intended for one-time use.
- Do not use the product if there is a doubt as to whether the product is sterile.
- Store in a dark, dry, cool place.
- Avoid extended exposure to light.
- Upon removal from package, inspect the product to ensure no damage has occurred.

### WARNING

- The device is intended to provide gentle debridement of the fistula tract to remove non-vascularized tissue.
- Care should be taken to not increase the diameter of the fistula tract.

### SUGGESTED INSTRUCTIONS FOR USE

1. Identify the primary and secondary fistula openings by inserting one end of the device into the most visible fistula opening.

**Note: If you are unable to identify both the primary and secondary openings, the use of a standard fistula probe may be necessary to facilitate brush placement.**

2. Navigate the device through the fistula tract.
3. The device should be advanced so that the brush portion is in contact with the fistula tract.
4. Gently clean/debride the fistula tract by moving the brush in a backwards and forwards motion. Removal of non-vascularized tissue from the fistula tract should be accomplished.
5. Flush the tract as thoroughly as possible with sterile saline, or comparable solution.
6. The suture tether should then be coupled to the Surgisis® Biodesign™ Recto-Vaginal/Anal Fistula Plug on the rectal side and pulled through the fistula tract, exiting at the second opening.

**Note: These instructions for use are based on experience from physicians and/or their published literature. Refer to your local Cook sales representative for information on available literature.**

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**COOK IRELAND LTD.**