Cook® IUI Catheters	EN 3
Katetry Cook® IUI	CS 3
Cook® katetre til intrauterin insemination	DA 4
Cook® IUI-Katheter	DE 5
Καθετήρες ενδομητρικής σπερματέγχυσης Cook®	EL 5
Catéteres de inseminación intrauterina Cook®	ES 6
Sondes pour IUI Cook®	FR 7
Cook® IUI (méhen belüli megtermékenyítő) katéterek	HU 8
Cateteri per IUI Cook®	IT 9
Cook® intra-uteriene inseminatiekatheters	NL 9
Cook® IUI-katetre	NO 10
Cewniki do zaplemnienia wewnątrzmacicznego Cook®	PL 11
Cateteres Cook® para inseminação intra-uterina	PT 12
Cook® IUI-katetrar	SV 13
Cook® 宫内授精导管	ZH 13

U _ J - C I C _ R E V . 1

COOK® IUI CATHETERS

Used for the introduction of washed spermatozoa into the uterine cavity. Cook® IUI catheters are designed to affix to a standard Luer-slip syringe. Some catheters are malleable and may be preformed to a desired shape. Intended for one-time use.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

This device should not be used on a patient with an active vaginal or intrauterine infection, a sexually transmitted disease, a recent uterine perforation, a recent cesarean section, is currently pregnant, or if the patient currently has, or suspects she may have an intrauterine device within her uterus.

CAUTION: This procedure should only be performed by physicians trained in intrauterine insemination techniques.

SUGGESTED INSTRUCTIONS FOR USING COOK® IUI CATHETERS

1. Form catheter to desired shape, if applicable.
2. Affix flared end or Luer-slip of catheter to syringe and load catheter with washed sperm solution.
3. Place patient in supine position with legs in stirrups.
4. Insert vaginal speculum to expose cervix.
5. Guide catheter through the cervix and into the uterus.
6. Contents are then instilled into the uterus.
7. Remove and discard syringe/catheter assembly.

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COOK IRELAND LTD.