**Laparoscopic Gamete Intrafallopian Transfer (GIFT)
Catheter Set**

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U _ K - J - L G C S _ R E V . 1

LAPAROSCOPIC GAMETE INTRAFALLOPIAN TRANSFER (GIFT) CATHETER SET

Used to transfer gametes directly into the fallopian tube under laparoscopic guidance. Intended for one-time use.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CONTRAINDICATIONS

This catheter set should only be used on patients with patent fallopian tube(s). This set should not be used on a patient with a sexually transmitted disease, a recent cesarean section, a recent pregnancy (or is currently pregnant), intrauterine device, intrauterine infection, a recent uterine perforation, tubal disease, or has had a tubal ectopic pregnancy.

PRECAUTIONS

The assisted reproduction catheter (and any other accessories used during this procedure) should be comprised of embryo compatible materials.

Infection may occur due to bacterial contamination. Recommendations to minimize occurrence include use of only embryo compatible materials, flushing the catheter (and any other accessories used) with sterile, compatible culture media, and closely adhering to sterile techniques.

Bleeding may occur as a result of trauma due to insertion of the catheter.

SUGGESTED INSTRUCTIONS FOR USING LAPAROSCOPIC GAMETE INTRAFALLOPIAN TRANSFER (GIFT) CATHETER SET

NOTE: It is important to determine the exact catheter volume prior to use. To determine catheter volume, aspirate transfer media or other appropriate liquid into the catheter, utilizing a calibrated syringe.

1. Nick the abdominal skin at desired position for ease of trocar entry.
2. Insert the trocar stylet/needle cannula assembly into the peritoneal cavity and remove stylet, leaving needle cannula in place.
3. Attach the endoscopic cap over the hub of the introducer. Insert the introducing/guide cannula or introducing/guide catheter through the needle cannula into the fallopian tube.
4. With the inner catheter pre-loaded with gametes, insert the inner catheter through the endoscopic cap/ introducing cannula or introducing catheter through the fallopian tube into the ampulla.
5. Expel the gametes.
6. Remove the device and discard.

NOTE: One cell mouse embryo tested and passed with 75% or greater blastocyst rate. USP endotoxin (LAL) tested and passed with 20 EU's or less per device. Testing is conducted on a lot-to-lot (batch) basis.

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COOK IRELAND LTD.