
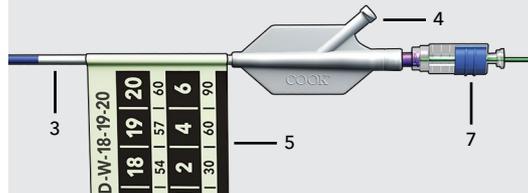
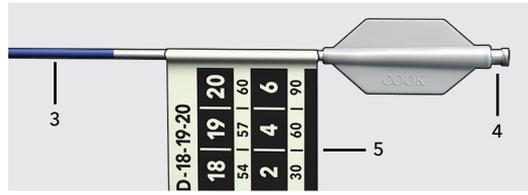
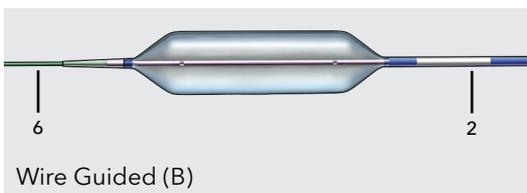


Hercules®

3 STAGE BALLOON

Features

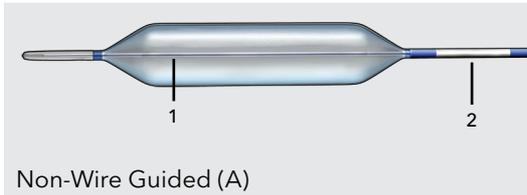

- 1) Rapid deflation sleeve
- 2) Silver positioning indicator
- 3) Express evacuation catheter
- 4) Inflation hub


- 5) Glow-in-the-dark catheter tag
- 6) 300 cm wire guide
- 7) Wire lock

Step 1a




Fill the 60 mL inflation device with water. Purge excess air and water from the syringe by tilting the syringe assembly so that air bubbles float toward the Luer lock end, then pump the handle level until all excess air and water is purged from syringe and connected tubing and the level of remaining water is 40 mL.

Step 1b

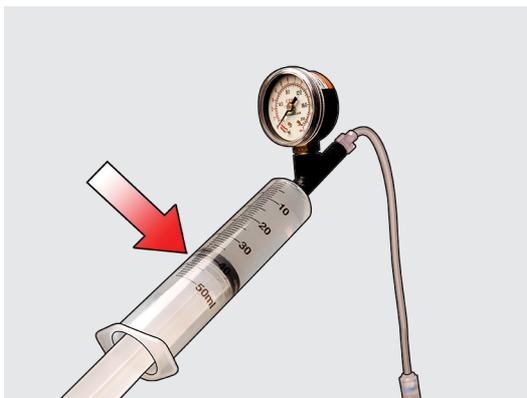


Maintain water level at the 40 mL line in the syringe as shown.

Step 2

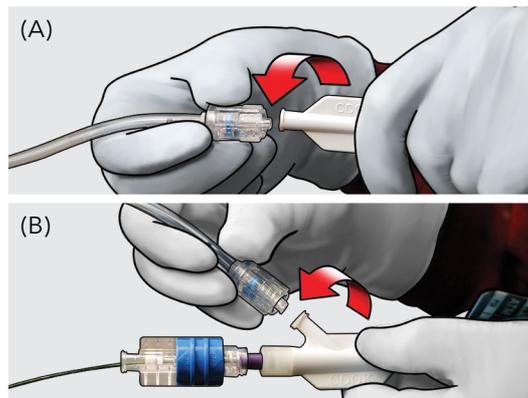


Securely attach the balloon inflation port to the inflation device.

Step 3

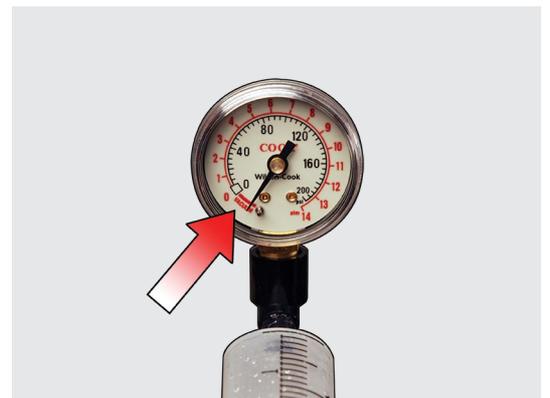


Apply negative pressure.

Step 4

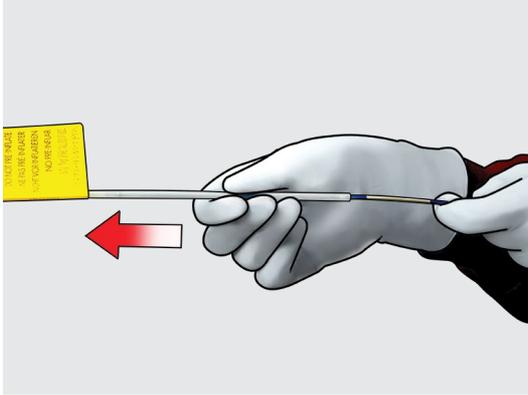


Remove the protective sheath from the balloon.

Step 5

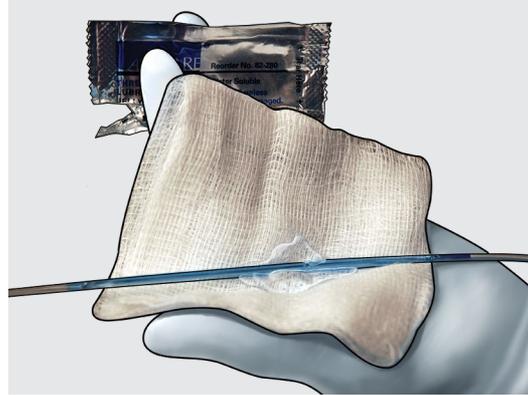


Lubricate the balloon completely. **Do not** pre-inflate the balloon prior to introduction into the endoscope. After visualization of the silver positioning indicator, advance the balloon to the desired position.

Step 5.5 (Optional)

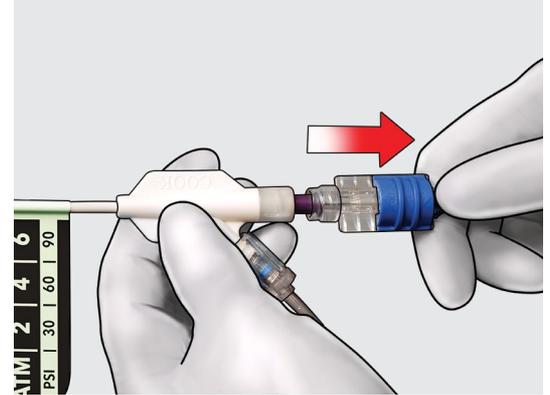

(Wire guided balloon only)
To advance the wire guide in the wire guided version, move the blue wire lock to the "ON" position.

Step 6

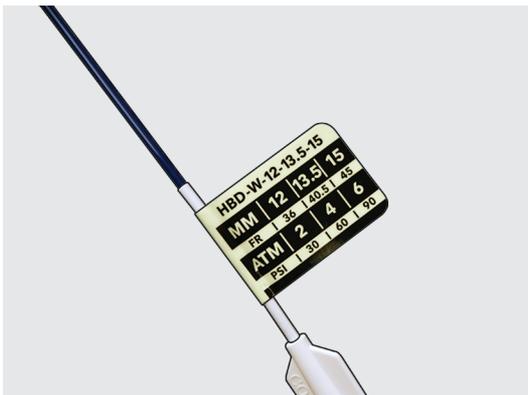

Increase pressure to the specified pressure levels shown on the catheter tag.

Step 7



For deflation, apply negative pressure. Remove the balloon from the scope after the balloon is deflated.

Reference Part Number	Order Number
HBD-8-9-10	G31928
HBD-10-11-12	G31925
HBD-12-13.5-15	G31926
HBD-15-16.5-18	G48732
HBD-18-19-20	G31927
HBD-W-8-9-10	G51799
HBD-W-10-11-12	G51795
HBD-W-12-13.5-15	G51796
HBD-W-15-16.5-18	G51797
HBD-W-18-19-20	G51798

Refer to the current Instructions for Use for detailed system use.

Customer Service

EU Website: cookmedical.eu
 EDI: cookmedical.eu/edi
 Distributors: +353 61239240, ssc.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001702, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 6950072804, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Iceland: +354 8007615, is.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Norway: +47 23162968, no.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com

USA Website: cookmedical.com
 EDI: cookmedical.com/edi.do
Americas:
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 E-mail: customersupport@cookmedical.com
Australia:
 Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283
 E-mail: cau.custserv@cookmedical.com



AI, ESC, IR, OHNS, PI, RH, SUR-8.5X11