
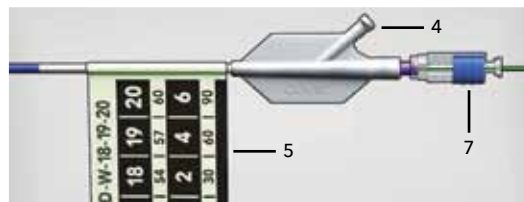
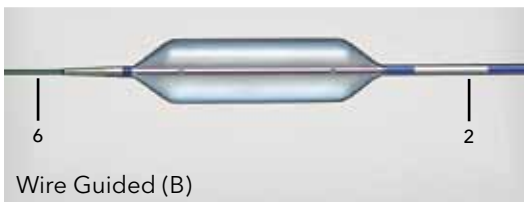


Hercules®

3 STAGE BALLOON

Features

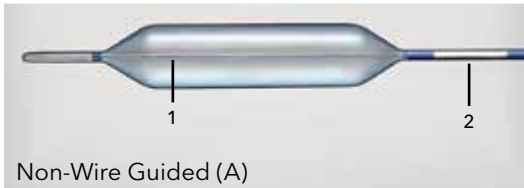

- 1) Rapid deflation* sleeve
- 2) Silver positioning indicator
- 3) Express evacuation catheter
- 4) Inflation hubs


- 5) Glow-in-the-dark catheter tags
- 6) 300 cm wire guide
- 7) Wire lock

Step 1a




Fill the 60 cc inflation device. Purge excess air and water from the syringe by tilting the syringe assembly so that air bubbles float toward the luer lock end, then pump the handle level until all excess air and water is purged from syringe and connected tubing to 40 cc.

Step 1b

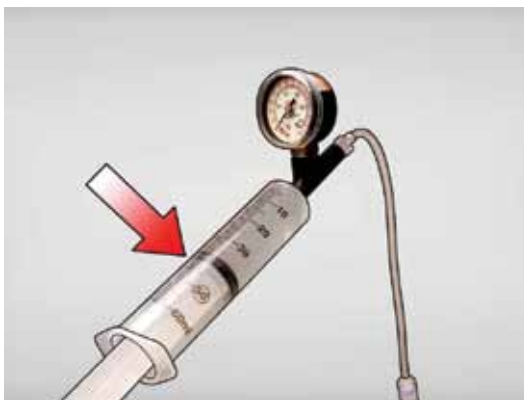

Maintain water level at the 40 cc line in the syringe as shown.

Step 2

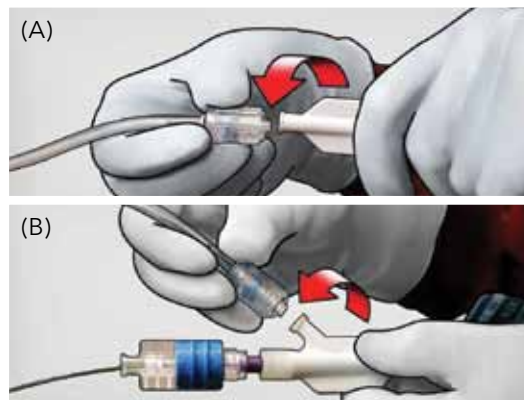


Securely attach balloon inflation port to inflation device.

Step 3




Apply negative pressure.

Step 4

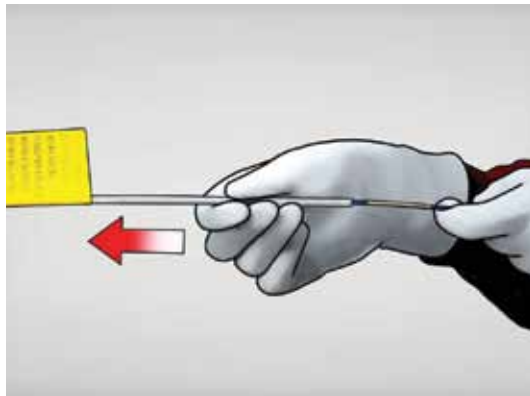


Remove protective sheath from balloon.

Step 5

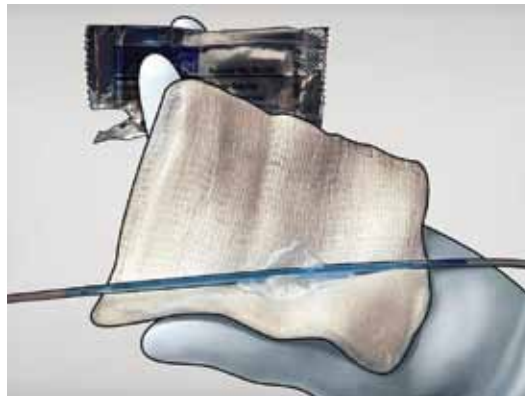


Lubricate balloon completely. **Do not** pre-inflate balloon, prior to introduction into the endoscope. After visualization of the silver positioning indicator, advance balloon to the desired position.

Step 6

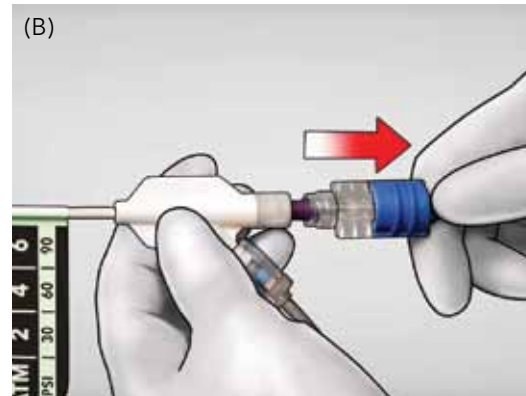

To advance the wire in the wire guided version, move the blue wire lock to the "ON" position.

Step 7

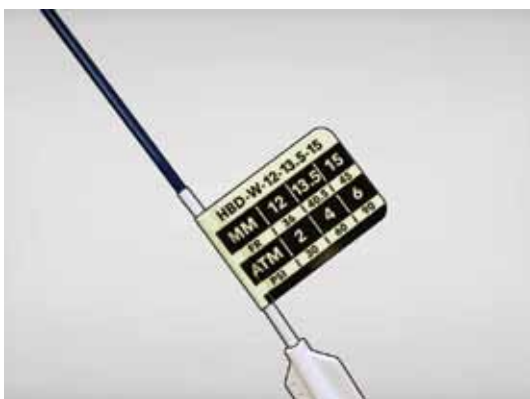

Increase pressure to the specified pressure levels shown on the catheter tag.

Step 8



For rapid deflation*, apply negative pressure. Remove balloon from scope after balloon is deflated.

Reference Part Number	Order Number
-----------------------	--------------

HBD-8-9-10	G31928
HBD-10-11-12	G31925
HBD-12-13.5-15	G31926
HBD-15-16.5-18	G48732
HBD-18-19-20	G31927
HBD-W-8-9-10	G51799
HBD-W-10-11-12	G51795
HBD-W-12-13.5-15	G51796
HBD-W-15-16.5-18	G51797
HBD-W-18-19-20	G51798

Refer to current instructions for detailed system use.
*Data on file at Cook Medical.

Customer Service

EMEA: EDI - www.cookmedical.com/edi.do
Distributors: +353 61239240, ssc.distributors@cookmedical.com
Austria: +43 179567121, oe.orders@cookmedical.com
Belgium: +32 27001633, be.orders@cookmedical.com
Denmark: +45 38487607, da.orders@cookmedical.com
Finland: +358 972519996, fi.orders@cookmedical.com
France: +33 171230269, fr.orders@cookmedical.com
Germany: +49 6950072804, de.orders@cookmedical.com
Hungary: +36 17779199, hu.orders@cookmedical.com
Ireland: +353 61239252, ie.orders@cookmedical.com
Italy: +39 0269682853, it.orders@cookmedical.com
Netherlands: +31 202013367, nl.orders@cookmedical.com
Norway: +47 23162968, no.orders@cookmedical.com
Spain: +34 912702691, es.orders@cookmedical.com
Sweden: +46 858769468, se.orders@cookmedical.com
Switzerland - French: +41 448009609, fr.orders@cookmedical.com
Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
Switzerland - German: +41 448009609, de.orders@cookmedical.com
United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 E-mail: orders@cookmedical.com

Australia:
 Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283
 E-mail: cau.custserv@cookmedical.com

