


Choose intuitive products
that were designed with
patients in mind.

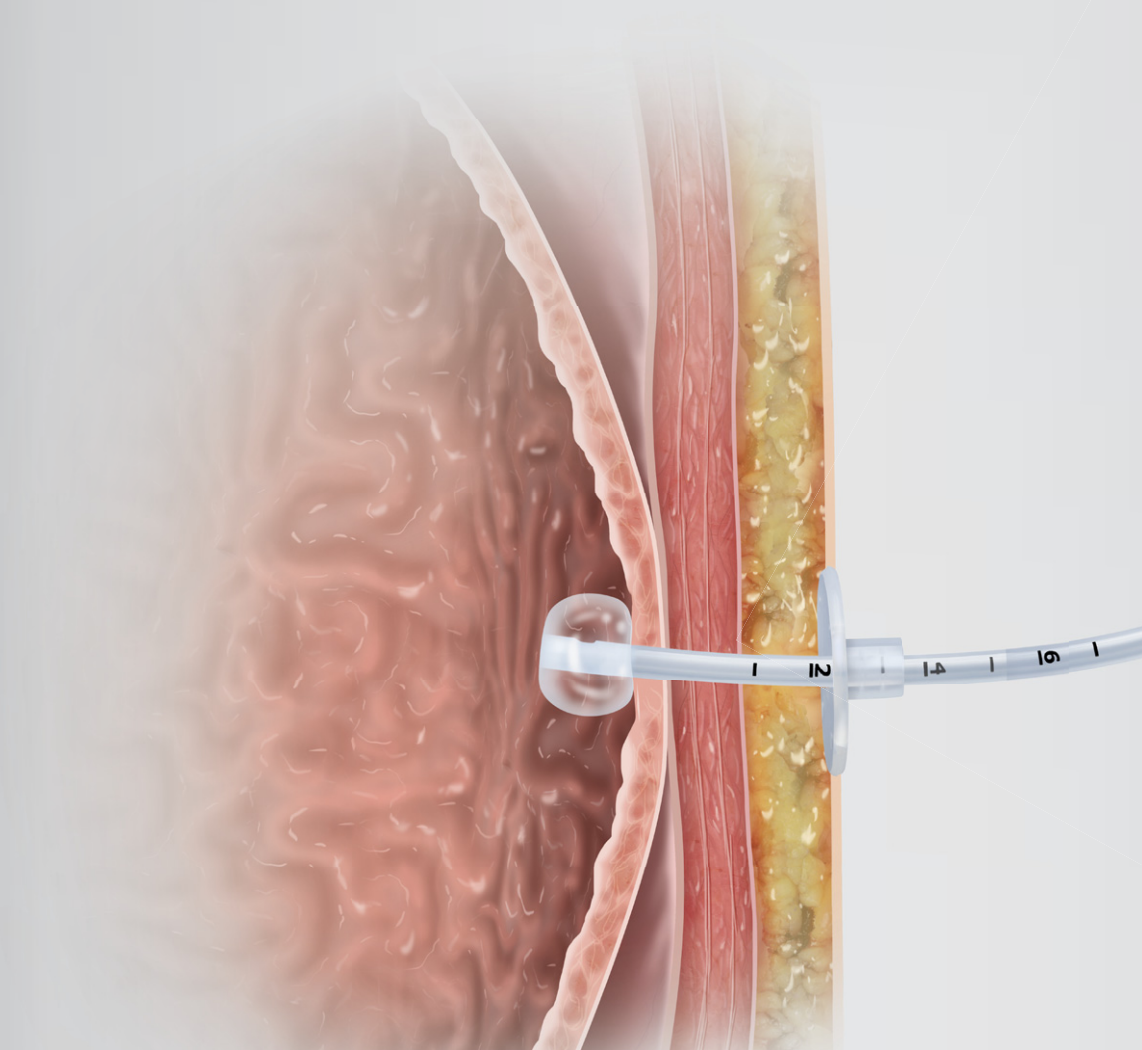

Entuit™

ENTERAL FEEDING PRODUCTS



www.cookmedical.com

Entuit Start

Simple for doctors.
Comfortable for patients.

Start with exactly what you need.

► **Entuit Start
Initial Placement Gastrostomy Set**

- Simple, intuitive arrangement
- Supports your procedure while minimizing product waste
- Sizes that work with the Entuit Thrive Balloon Retention Gastrostomy Tube




Entuit Secure

Choose from two suture anchor sets, one of which allows tension adjustment after placement.

► Entuit Secure Suture Anchor Sets

- Each suture anchor is preloaded in the needle and is available in sets of two or three.
- A unique, patented locking mechanism easily secures the suture.
- The innovative, tension-adjustable suture anchor set is available to help increase patient comfort.

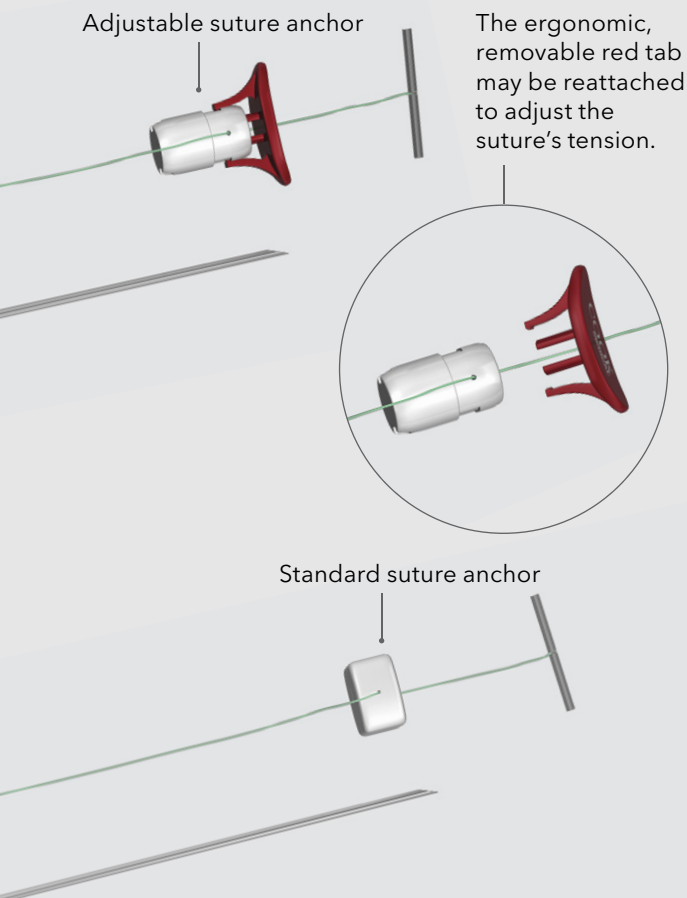


Entuit Thrive

Use the feeding tube with an in-line balloon shaft that's easy to place and comfortable for your patient.

► Entuit Thrive Balloon Retention Gastrostomy Feeding Tube

- Slim, smooth, in-line balloon shaft
- Durable, longer-lasting balloon*
- Sizes that work with the Entuit Start Initial Placement Gastrostomy Set

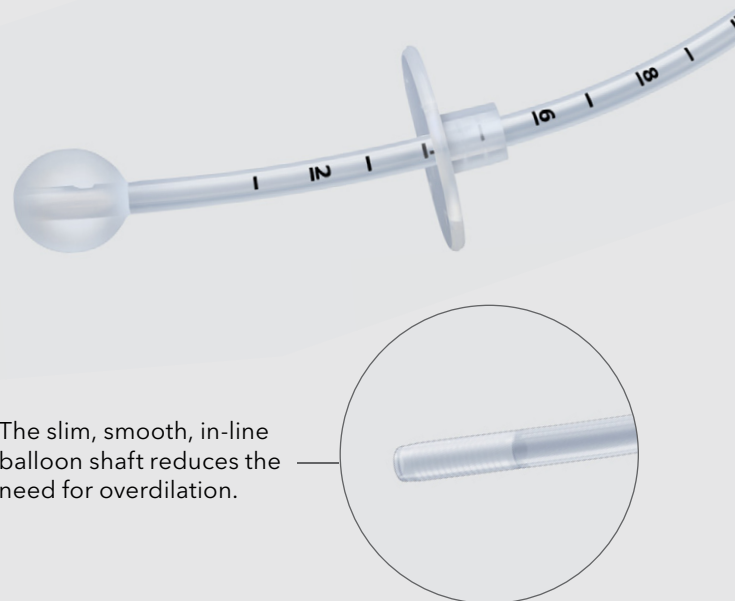

*Data on file, based on durability testing in simulated gastric acid using ASTM International methodology.

Start with exactly what you need.




Entuit Start Initial Placement Gastrostomy Set

- The set includes the components that are essential to initially placing a feeding tube. To simplify the procedure, the components are intuitively arranged in the order in which they are used.
- To provide greater purchasing flexibility and to minimize product waste, the Entuit Secure suture anchors and Entuit Thrive feeding tube are not included in the set. They can be ordered separately.
- The sets are available in sizes (small, medium, or large) that are appropriate for your specific tube.
- The tapered Coons dilator minimizes the need for extra dilators and helps ensure proper dilation.



SCALPEL

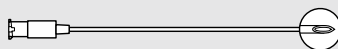


ONE-PART PERCUTANEOUS ENTRY NEEDLE
18 gage, stainless steel


AMPLATZ EXTRA-STIFF WIRE GUIDE
.038 inch diameter, 100 cm long



SLIP TIP SYRINGE
6 mL

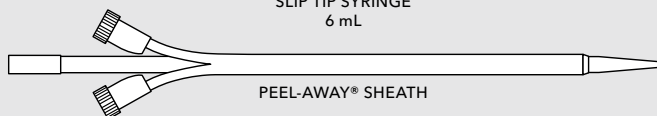

PEEL-AWAY® SHEATH

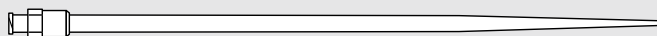

COONS DILATORS
Hydrophilic coating

Order Number	Reference Part Number	Set Size	Used with Entuit Thrive Feeding Tube* Fr	Dilator Fr	Sheath Fr
G34442	IPSG-S**	small	12, 14	12, 14	16
G34443	IPSG-M†	medium	16, 18	16, 18	18, 20
G34444	IPSG-L‡	large	20, 22, 24	20, 22, 24	22, 26

*Feeding tubes are not included in Entuit Start Initial Placement Gastrostomy Sets.


**Includes two dilators and one sheath.

†Includes two dilators and two sheaths.

‡Includes three dilators and two sheaths.


Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Adjust the suture anchor tension even after placement.

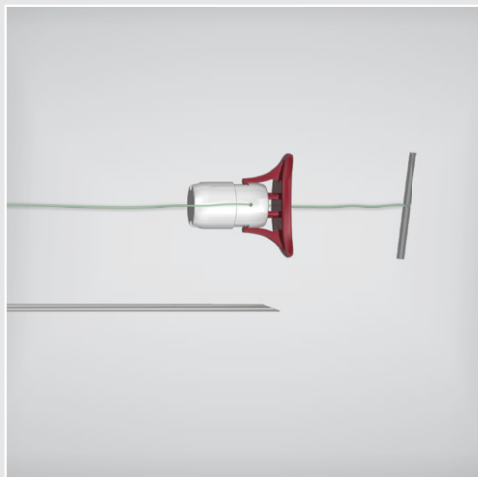

Entuit Secure Adjustable Gastrointestinal Suture Anchor Set


- The unique, patented, adjustable locking mechanism allows you to increase tension during critical points in the gastropexy procedure and decrease the tension later, if needed, to make patients comfortable.
- After you achieve optimal tension, remove the red tab to lock the suture anchor in place. To adjust the tension, you can reattach the red tab.
- The smooth, flexible “spring coil” tips of the suture anchor lie comfortably contoured against the stomach wall.
- Each suture anchor is preloaded in the needle. Suture anchors can be ordered in sets of two or three.
- Sutures are made of strong, braided material.

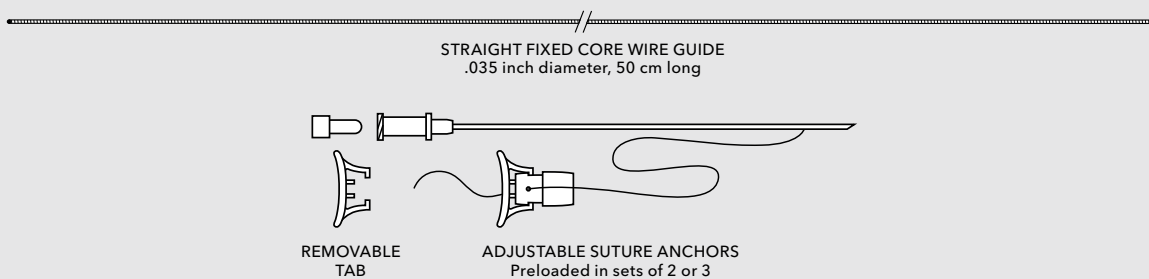

Order Number	Reference Part Number	Preloaded Suture Anchors	Needle Length cm	Needle gage
G35562	GIAS-SRM-ADJ-2	2	12	17
G35563	GIAS-SRM-ADJ-3	3	12	17


Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Perform an efficient gastropexy.

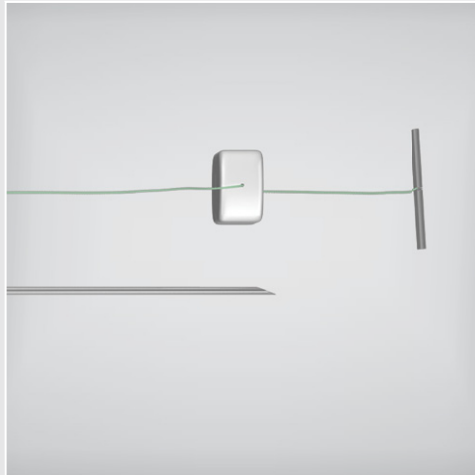

Entuit Secure Gastrointestinal Suture Anchor Set

- Sutures are made of strong, braided material.
- The smooth, flexible “spring coil” tips of the suture anchor lie comfortably contoured against the stomach wall.
- The locking suture anchor can be quickly tightened to easily secure the stomach wall to the abdomen.
- The unique, patented locking mechanism allows you to maintain tension while you tighten the suture.
- Each suture anchor is preloaded in the needle. Suture anchors can be ordered in sets of two or three.

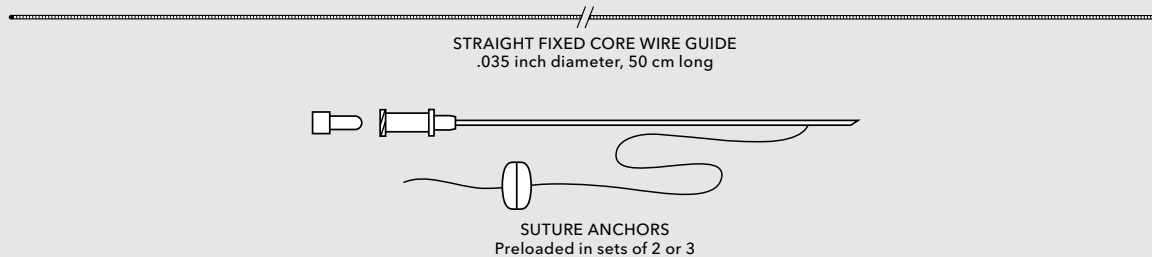

Order Number	Reference Part Number	Preloaded Suture Anchors	Needle Length cm	Needle gage
G35569	GIAS-SRM-2	2	12	17
G35570	GIAS-SRM-3	3	12	17

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.


Choose the slim, smooth, in-line balloon shaft that is easy to place and comfortable for your patient.



Entuit Thrive Balloon Retention Gastrostomy Feeding Tube

- The unique, integrated design of the in-line balloon shaft on 14 Fr and larger tubes reduces the need for overdilation during both placement and replacement procedures.
- The durable balloon on the Entuit Thrive standard-profile feeding tube lasts 20.5 times longer than a leading tube provider's standard-profile balloon.*
- The feeding tube sizes match perfectly with the Entuit Initial Placement Gastrostomy Sets to streamline placement procedures.
- The medicine port's unique lid accommodates a 60 mL syringe and an oral syringe, to make feeding more convenient for patients.

*Data on file, based on durability testing in simulated gastric acid using ASTM International methodology.

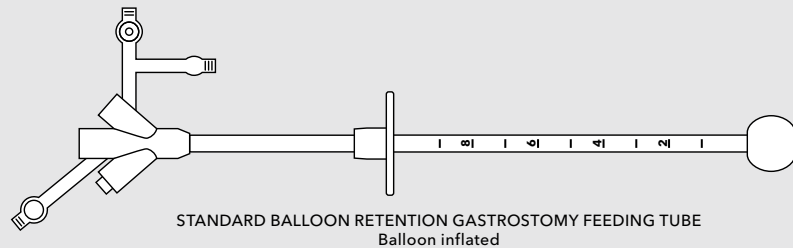

Order Number	Reference Part Number	Fr	Marked Length cm	Balloon Volume mL	Used with Entuit Start Initial Placement Gastrostomy Set** size
G35614	SBRD-12	12	10	5	small
G35615	SBRD-14	14	10	5	small
G35616	SBRD-16	16	10	5	medium
G35617	SBRD-18	18	10	5	medium
G35618	SBRD-20	20	10	20	large
G35619	SBRD-22	22	10	20	large
G35620	SBRD-24	24	10	20	large

**Must be ordered separately.


Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Help your patients succeed.

Please tell your patients about the Entuit Thrive patient guide (pictured below), which provides useful information about living with a feeding tube, including feeding instructions, feeding tube maintenance guidelines, and online resources.

You may request copies of the patient guide from your Cook representative and give them to your patients.

Patients and caregivers can find information about using and caring for a feeding tube and view a patient video at www.cookfeedingtube.com.

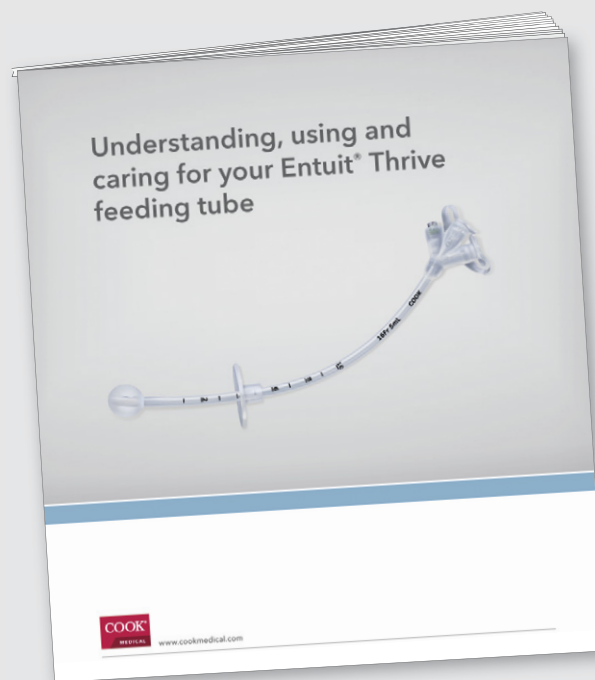

Customer Service

EMEA: EDI - www.cookmedical.com/edi.do
Distributors: +353 61239240, ssc.distributors@cookmedical.com
Austria: +43 179567121, oe.orders@cookmedical.com
Belgium: +32 27001633, be.orders@cookmedical.com
Denmark: +45 38487607, da.orders@cookmedical.com
Finland: +358 972519996, fi.orders@cookmedical.com
France: +33 171230269, fr.orders@cookmedical.com
Germany: +49 6950072804, de.orders@cookmedical.com
Hungary: +36 17779199, hu.orders@cookmedical.com
Ireland: +353 61239252, ie.orders@cookmedical.com
Italy: +39 0269682853, it.orders@cookmedical.com
Netherlands: +31 202013367, nl.orders@cookmedical.com
Norway: +47 23162968, no.orders@cookmedical.com
Spain: +34 912702691, es.orders@cookmedical.com
Sweden: +46 858769468, se.orders@cookmedical.com
Switzerland - French: +41 448009609, fr.orders@cookmedical.com
Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
Switzerland - German: +41 448009609, de.orders@cookmedical.com
United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
E-mail: orders@cookmedical.com

Australia:
Phone: +61 734346000, 1800777222, Fax: +61 738411288, 1800077283
E-mail: cau.custserv@cookmedical.com

